
# Adhesive Tabs

For use with Masimo® Disposable Sensors





Images	2
<b>en</b> English	3
<b>fr</b> French	4
<b>de</b> German	5
<b>it</b> Italian	6
<b>es</b> Spanish	7
<b>sv</b> Swedish	8
<b>nl</b> Dutch	9
<b>da</b> Danish	10
<b>pt</b> Portuguese	11
<b>zh</b> Chinese	12
<b>ja</b> Japanese	13
<b>fi</b> Finnish	14
<b>no</b> Norwegian	15
<b>cs</b> Czech	16
<b>hu</b> Hungarian	17
<b>pl</b> Polish	18
<b>ro</b> Romanian	19
<b>sk</b> Slovak	20
<b>tr</b> Turkish	21
<b>el</b> Greek	22
<b>ru</b> Russian	23
<b>ar</b> Arabic	24
<b>fa</b> Farsi	25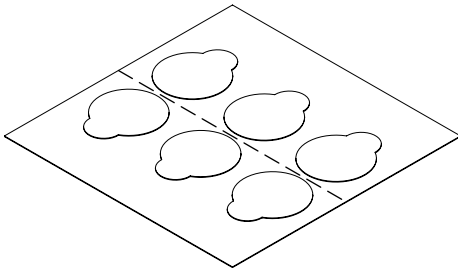

# Adhesive Tabs

For use with Masimo® Disposable Sensors

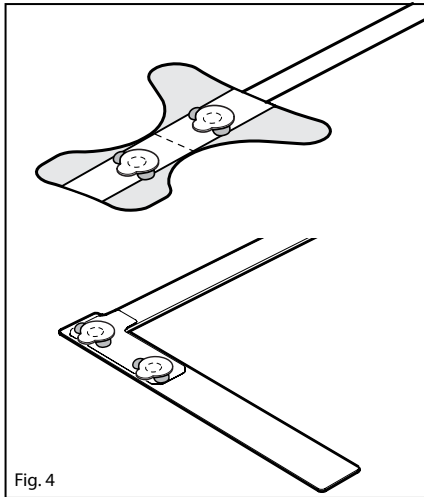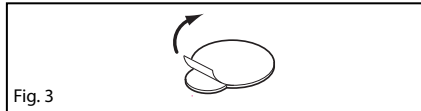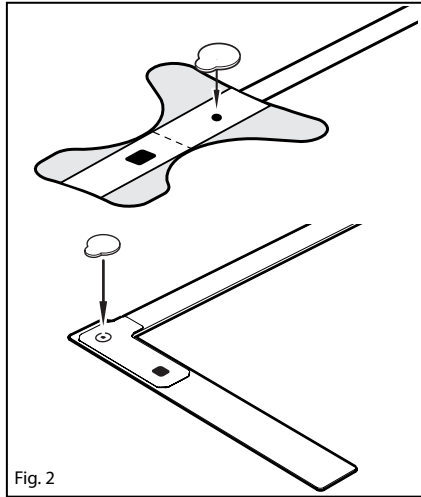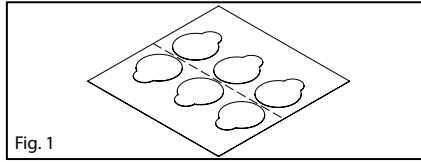# Adhesive Tabs

en

## For use with Masimo® Disposable Sensors

### DIRECTIONS FOR USE

 Single patient use only

 Not made with natural rubber latex

 Non-sterile

Prior to use, the user should read and understand the Operator's Manual for the device and the sensors.

#### DESCRIPTION

The disposable sensor adhesive tabs are double sided adhesive tabs used when the stickiness of the adhesive covering the optical components of the disposable adhesive sensors are no longer effective.

#### INSTRUCTIONS

- Refer to Fig. 1.** Remove the adhesive tab from the release liner by pulling up on the tab.
- Refer to Fig. 2.** Place an adhesive tab over each window of the sensor with the white area outside the optical window area.
- Refer to Fig. 3.** Remove the protective paper that covers each tab and reapply the sensor to the same patient.
- Refer to Fig. 4.** When the adhesive on the first set of tabs is no longer sticky, a second set may be applied. Up to 3 sets of adhesive tabs may be applied to each window, placing one on top of the other.

**Note:** If the adhesive no longer adheres to the skin, use a new sensor.

The following symbols may appear on the product or product labeling:

SYMBOL	DEFINITION	SYMBOL	DEFINITION	SYMBOL	DEFINITION
	Follow instructions for use		Not made with natural rubber latex	<b>Rx ONLY</b>	<b>Caution:</b> Federal law (USA) restricts this device to sale by or on the order of a physician
	Consult instructions for use	<b>LOT</b>	Lot code	<b>CE</b>	Mark of conformity to European Medical Device Directive 93/4/EEC
	Manufacturer	<b>REF</b>	Catalogue number (model number)	<b>EC REP</b>	Authorized representative in the European community
	Date of manufacture YYYY-MM-DD	<b>####</b>	Masimo reference number		Body weight
	Use by YYYY-MM-DD		Storage humidity limitation		Storage temperature range
	Do not re-use/Single patient use only		Do not use if package is damaged		Keep dry
	Non-Sterile		Instructions/Directions for Use/Manuals are available in electronic format @ <a href="http://www.Masimo.com/TechDocs">http://www.Masimo.com/TechDocs</a> <b>Note: eFU is not available for CE mark countries.</b>		

Patents: <http://www.masimo.com/patents.htm>

Masimo, SET, , and rainbow are federally registered trademarks of Masimo Corporation.

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© 2016 Masimo Corporation
EU Authorized Representative for Masimo Corporation:
MDSS GmbH