
RD SET™




SofTouch™ Replacement Wraps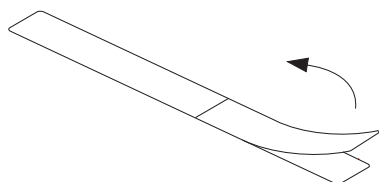

Images	2
en English	3
fr French	4
de German	5
it Italian	6
es-i Spanish (International)	7
sv Swedish	8
nl Dutch	9
da Danish	10
pt-p Portuguese (Portugal)	11
zh-s Chinese (Simplified)	12
ja Japanese	13
fi Finnish	14
no Norwegian	15
cs Czech	16
hu Hungarian	17
pl Polish	18
ro Romanian	19
sk Slovak	20
tr Turkish	21
el Greek	22
ru Russian	23
ar Arabic	24
fa Farsi	25

RD SET™

SofTouch™ Replacement Wraps

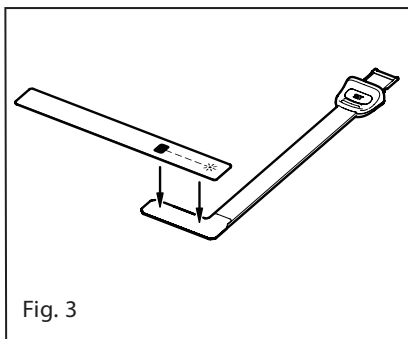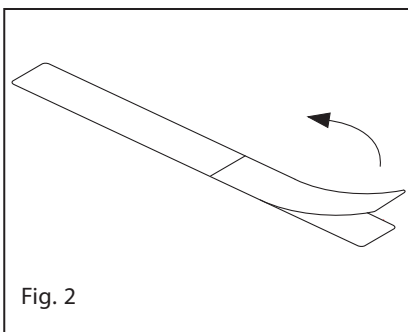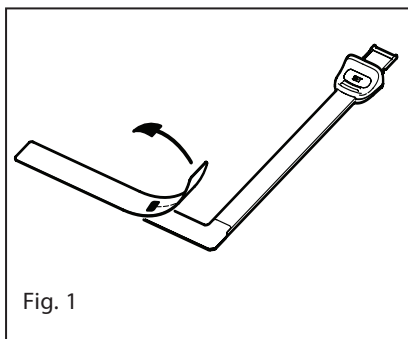RD SET™

SofTouch™ Replacement Wraps

en

DIRECTIONS FOR USE

② Single patient use only

 Not made with natural rubber latex

 Non-Sterile

DESCRIPTION

The RD SET™ SofTouch™ replacement wraps are used to replace the attachment wrap portion of the RD SET disposable sensors in the event that the wrap becomes soiled or torn prior to the end of sensor life.

INSTRUCTIONS

1. Remove the existing wrap and discard (Fig. 1).
2. Remove the backing from the wrap (Fig. 2).
3. Position the replacement wrap over the sensor, aligning the emitter component with the sensor cable. (Fig. 3).

The following symbols may appear on the product or product labeling:

SYMBOL	DEFINITION	SYMBOL	DEFINITION
	Consult instructions for use		Mark of conformity to European medical device directive 93/42/EEC
 (blue background)	Follow instructions for use		Authorized representative in the European community
	Manufacturer		Lot code
	Use-by date YYYY-MM-DD		Catalogue number (model number)
	Do not re-use/Single patient use only		Pediatric patient
	Non-Sterile		Storage humidity limitation
	Not made with natural rubber latex		Storage temperature range
	Do not use if package is damaged		Keep dry
Rx ONLY	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.		Instructions/Directions for Use/Manuals are available in electronic format @ http://www.Masimo.com/TechDocs Note: elFU is not available for CE mark countries.
	Masimo reference number		Atmospheric pressure limitation

RD SET is a trademark of Masimo Corporation.

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© 2016 Masimo Corporation
MDSS GmbH