
RD SET™




Adhesive Replacement Tapes

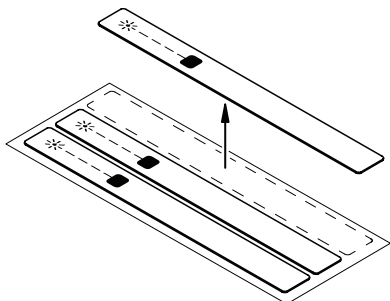| | |
|-------------------------------------|----|
| Images | 2 |
| en English | 3 |
| fr French | 4 |
| de German | 5 |
| it Italian | 6 |
| es-i Spanish (International) | 7 |
| sv Swedish | 8 |
| nl Dutch | 9 |
| da Danish | 10 |
| pt-p Portuguese (Portugal) | 11 |
| zh-s Chinese (Simplified) | 12 |
| ja Japanese | 13 |
| fi Finnish | 14 |
| no Norwegian | 15 |
| cs Czech | 16 |
| hu Hungarian | 17 |
| pl Polish | 18 |
| ro Romanian | 19 |
| sk Slovak | 20 |
| tr Turkish | 21 |
| el Greek | 22 |
| ru Russian | 23 |
| ar Arabic | 24 |
| fa Farsi | 25 |

RD SET™

Adhesive Replacement Tapes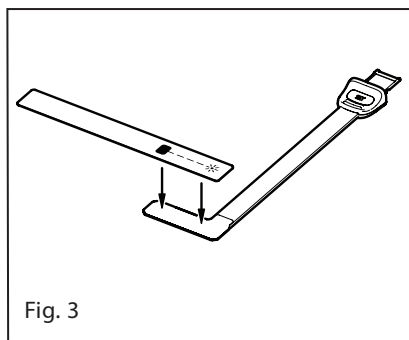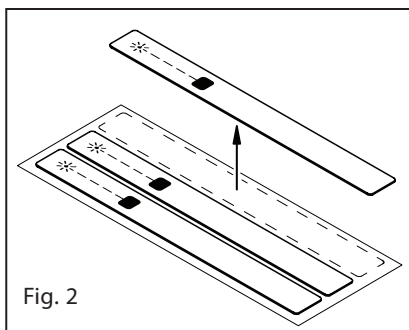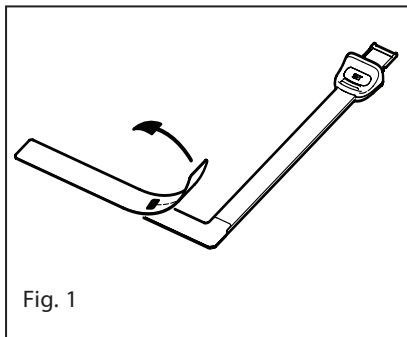

Adhesive Replacement Tapes

DIRECTIONS FOR USE

 Single Patient Use only

 Not made with natural rubber latex

 Non-Sterile

DESCRIPTION

The RD SET™ adhesive replacement tapes are used to replace the attachment tape portion of the RD SET disposable sensors in the event that the tape becomes soiled or torn prior to the end of the sensor life.

INSTRUCTIONS

1. Remove the existing tape and discard (Fig. 1).
2. Remove the replacement tape from the release liner (Fig. 2).
3. Position the replacement tape over the sensor, aligning the emitter component with the sensor cable (Fig. 3).

The following symbols may appear on the product or product labeling:

| SYMBOL | DEFINITION | SYMBOL | DEFINITION |
|--|---|---|---|
|  | Consult Instructions for Use |  | Mark of Conformity to European Medical Device Directive 93/42/EEC |
|  (blue background) | Follow Instructions for Use |  | Authorized representative in the European community |
|  | Manufacturer |  | Lot code |
|  | Use By YYYY-MM |  | Catalogue number (model number) |
|  | Single Patient Use only |  | Pediatric patient |
|  | Non-Sterile |  | Storage Humidity Limitation |
|  | Not made with natural rubber latex |  | Storage Temperature Limitation |
|  | Do not use if package is damaged |  | Keep Dry |
| Rx Only | Federal law (USA) restricts this device to sale by or on the order of a physician |  | Instructions/Directions for Use/Manuals are available in electronic format @ http://www.Masimo.com/TechDocs Note: eIFU is not available for CE mark countries. |

RD SET is a trademark of Masimo Corporation.

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© 2015 Masimo Corporation
MDSS GmbH