

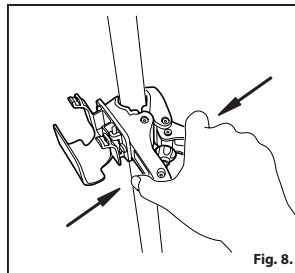
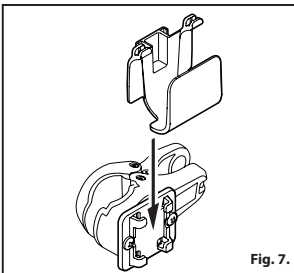
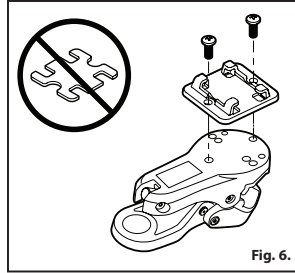
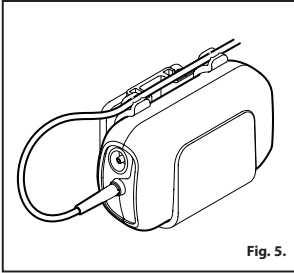
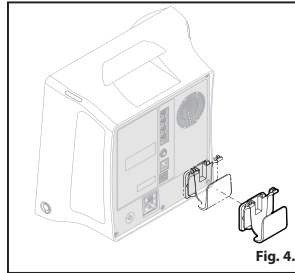
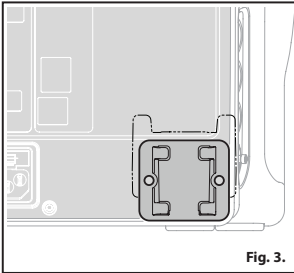
NomoLine® ISA™ Mounting Kit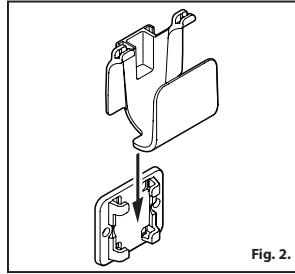

Images	2
en English	3-4
fr French	5-6
de German	7-8
it Italian	9-10
es Spanish	11-12
sv Swedish	13-14
nl Dutch	15-16
da Danish	17-18
pt Portuguese	19-20
zh Chinese	21-22
ja Japanese	23-24
fi Finnish	25-26
no Norwegian	27-28
cs Czech	29-30
hu Hungarian	31-32
pl Polish	33-34
ro Romanian	35-36
sk Slovak	37-38
tr Turkish	39-40
el Greek	41-42
ru Russian	43-44
ko Korean	45-46
et Estonian	47-48
lt Lithuanian	49-50
lv Latvian	51-52
ar Arabic	54-53

NomoLine® ISA™ Mounting Kit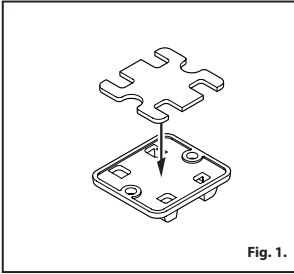

Mounting Kit

DIRECTIONS FOR USE

INTENDED USE

The NomoLine® ISA™, Mounting Kit is used to attach the NomoLine ISA Gas Analyzer to a host device or monitor. (Such as the Masimo® Root® device).

DESCRIPTION

The NomoLine ISA, Mounting Kit is an accessory for use with a host device/monitor and includes:

- (1) Mounting Plate
- (1) Mounting Bracket
- (1) Adhesive Tape

INSTRUCTIONS

Mount the NomoLine ISA gas analyzer to a host device/monitor:

1. Peel off the backing from the adhesive tape.
2. Refer to **Fig. 1**. Attach the adhesive tape to the mounting plate.
3. Refer to **Fig. 2**. Slide the bracket into the mounting plate.
4. Peel off the release liner.
5. Refer to **Fig. 3**. Verify placement of the bracket on the host device/monitor. (Such as the Masimo Root device - shown).
6. Refer to **Fig. 4**. Attach the assembled bracket and plate to the host device/monitor. (Such as the Masimo Root device - shown).
7. Refer to **Fig. 5**. Slide the NomoLine ISA gas analyzer into the mounted bracket.

Mount the NomoLine ISA gas analyzer to a pole clamp:

1. Do not attach the adhesive onto the mounting plate.
2. Refer to **Fig. 6**. Screw the mounting plate onto the pole clamp using the screws provided.
3. Refer to **Fig. 7**. Slide the bracket into the mounting plate.
4. Refer to **Fig. 8**. Attach the clamp onto the pole.
5. Refer to **Fig. 5**. Slide the NomoLine ISA gas analyzer into the mounted bracket.

WARNINGS

- The adhesive cannot be removed or repositioned once it adheres to the host device/monitor.
- Do not pick up the host device/monitor from the bracket.
- The ISA analyzer must be properly positioned in mounting bracket before use.
- Do not use the ISA analyzer device with mounting accessory if mounting plate or bracket are damaged.
- The mounting plate and bracket must be properly mounted to external equipment before use.

CLEANING

To clean use a soft cloth wipe each of the parts with the following chemicals for 30 seconds.

Cleaning solutions

- 70% isopropyl alcohol
- Glutaraldehyde
- 10% bleach (sodium hypochlorite) to 90% water solution
- Quaternary ammonium chloride

CAUTION

- Do not immerse in any liquid solution.
- Do not sterilize by irradiation, steam, autoclave, or ethylene oxide.
- Do not clean with chemicals not approved above.

ENVIRONMENTAL

Storage Temperature	-40°F to 158°F (-40°C to 70°C)
Relative Humidity	10% to 95% non-condensing

DISPOSAL

Dispose the used product at the end of the expected service life in accordance with local regulations.

WARRANTY

Masimo warrants to the initial buyer only that these products, when used in accordance with the directions provided with the Products by Masimo, will be free of defects in materials and workmanship for a period of six (6) months. Single use

products are warranted for single patient use only.

THE FOREGOING IS THE SOLE AND EXCLUSIVE WARRANTY APPLICABLE TO THE PRODUCTS SOLD BY MASIMO TO BUYER. MASIMO EXPRESSLY DISCLAIMS ALL OTHER ORAL, EXPRESS OR IMPLIED WARRANTIES, INCLUDING WITHOUT LIMITATION ANY WARRANTIES OF MERCHANTABILITY OR FITNESS FOR PARTICULAR PURPOSE. MASIMO'S SOLE OBLIGATION AND BUYER'S EXCLUSIVE REMEDY FOR BREACH OF ANY WARRANTY SHALL BE, AT MASIMO'S OPTION, TO REPAIR OR REPLACE THE PRODUCT.

WARRANTY EXCLUSIONS

This warranty does not extend to any product that has been used in violation of the operating instructions supplied with the product, or has been subject to misuse, neglect, accident or externally created damage. This warranty does not extend to any product that has been connected to any unintended device or system, has been modified, or has been disassembled or reassembled. This warranty does not extend to products that have been reprocessed, reconditioned or recycled.

IN NO EVENT SHALL MASIMO BE LIABLE TO BUYER OR ANY OTHER PERSON FOR ANY INCIDENTAL, INDIRECT, SPECIAL OR CONSEQUENTIAL DAMAGES (INCLUDING WITHOUT LIMITATION LOST PROFITS), EVEN IF ADVISED OF THE POSSIBILITY THEREOF. IN NO EVENT SHALL MASIMO'S LIABILITY ARISING FROM ANY PRODUCTS SOLD TO BUYER (UNDER A CONTRACT, WARRANTY, TORT OR OTHER CLAIM) EXCEED THE AMOUNT PAID BY BUYER FOR THE LOT OF PRODUCT(S) INVOLVED IN SUCH CLAIM. IN NO EVENT SHALL MASIMO BE LIABLE FOR ANY DAMAGES ASSOCIATED A PRODUCT THAT HAS BEEN REPROCESSED, RECONDITIONED OR RECYCLED. THE LIMITATIONS IN THIS SECTION SHALL NOT BE DEEMED TO PRECLUDE ANY LIABILITY THAT, UNDER APPLICABLE PRODUCTS LIABILITY LAW, CANNOT LEGALLY BE PRECLUDED BY CONTRACT.

NO IMPLIED LICENSE

Purchase or possession of this product does not carry any express or implied license to use with any device that is not an authorized device or separately authorized to use Masimo products.

If you encounter any serious incident with product, please notify the competent authority in your country and the manufacturer.

The following symbols may appear on the product or product labeling:

SYMBOL	DEFINITION	SYMBOL	DEFINITION	SYMBOL	DEFINITION
	Follow instructions for use		Gas sample	Rx ONLY	Caution: Federal law (USA) restricts this device to sale by or on the order of a physician
	Consult instructions for use		Lot code		Authorized representative in the European community
	Manufacturer		Catalogue number (model number)		Body weight
	Use-by YYYY-MM-DD		Masimo reference number		Keep dry
	Do not re-use/Single patient use only		Greater than		Do not use if package is damaged and consult instructions for use
	Not made with natural rubber latex		Less than		Storage temperature range
	Caution		Storage humidity limitation		Single patient - multiple use
	Mark of Conformity to European Medical Device Directive 93/42/EEC		Atmospheric pressure limitation		Distributor
	Medical device		Unique device identifier		Importer
	Instructions/Directions for Use/Manuals are available in electronic format @ http://www.Masimo.com/TechDocs Note: eIFU is not available in all countries.				

Patents: <http://www.masimo.com/patents.htm>.

ISA is a trademark of Masimo Corporation.

Masimo, Root, and NomoLine are federally registered trademarks of Masimo Corporation.

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© 2021 Masimo Corporation