Rad-G®

Battery Replacement Kit



Images	2
en English	3-4
fr French	5-6
de German	7-8
it Italian	9-10
es Spanish	11-12
pt-p Iberian Portuguese	13-14
pl Polish	15-16
sv Swedish	17-18
nl Dutch	19-20
da Danish	21-22
fi Finnish	23-24
no Norwegian	25-26
pt-b Brazilian Portuguese	27-28
tr Turkish	29-30
ru Russian	31-32
ko Korean	33-34
cs Czech	35-36
el Greek	37-38
sk Slovak	39-40
ro Romanian	41-42

Rad-G®

Battery Replacement Kit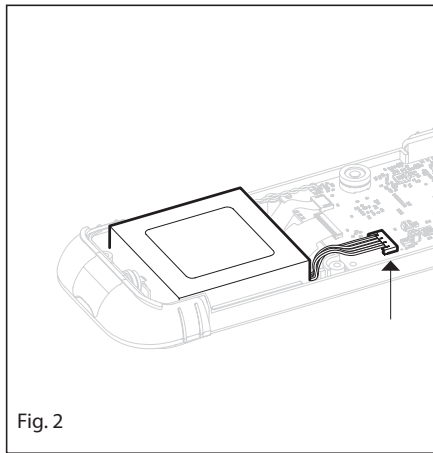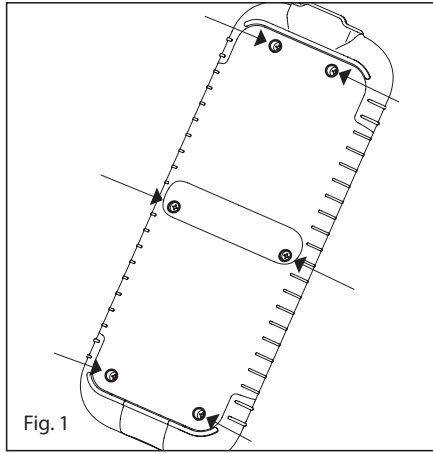

Rad-G®

Battery Replacement Kit

DIRECTIONS FOR USE

The Rad-G Li-Ion Battery Replacement Kit PN 5148 includes the following:

- Battery - 27058

CAUTIONS

- Only use the battery (PN 27058) included in the kit. Use of other batteries may affect performance.

INSTRUCTIONS

1. Ensure the Rad-G is turned off. Press and hold the power button until the device beeps once and shuts off.
2. Loosen the six (6) screws using a small Philips screwdriver on the back cover (Fig. 1) and remove the cover.
3. Disconnect the battery connector (Fig. 2) by gently tugging on the male six pin connector and remove the battery.
4. Place the new battery in the Rad-G device and connect the male six pin connector into the appropriate female pin connector port.
Note: Route the battery wiring exactly as the original wiring was routed to prevent the wiring from being pinched between the device and the back cover.
5. Install the back cover. Install and tighten the screws using a small Philips screwdriver.
6. Charge the device and power on to verify battery has been properly installed.

ENVIRONMENTAL

Storage Humidity	10% – 80% non-condensing
Operating Temperature	0°C – 45°C (32°F – 113°F) at ambient humidity
Storage Temperature	1 Month -10°C – +55°C (14°F – 131°F) 6 Month -10°C – +45°C (14°F – 113°F) 12 Month -10°C – +25°C (14°F – 77°F)

CAUTION: FEDERAL LAW (U.S.A.) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN.

If you encounter any serious incident with product, please notify the competent authority in your country and the manufacturer.

The following symbols may appear on the product or product labeling:

SYMBOL	DEFINITION	SYMBOL	DEFINITION
	Follow instructions for use		Separate collection for electrical and electronic equipment (WEEE).
	Consult instructions for use		Lot code
	Manufacturer		Catalogue number (model number)
	Date of manufacture YYYY-MM-DD		Masimo reference number
	Use -by YYYY-MM-DD		Fragile, handle with care
	Do not re-use/Single patient use only		Greater than
	Non-Sterile		Less than
	Not made with natural rubber latex		Caution
	Single patient - multiple use		Unique device identifier
	Distributor		Medical device
Rx ONLY	Caution: Federal law (USA) restricts this device to sale by or on the order of a physician		Keep dry

	European Conformity Mark		Do not use if package is damaged and consult instructions for use
	Authorized representative in the European community		Atmospheric pressure limitation
	Body weight		Storage humidity limitation
	Storage temperature range		Importer
	Instructions/Directions for Use/Manuals are available in electronic format @ http://www.Masimo.com/TechDocs Note: eFU is not available in all countries.		

Rad-G is a federally registered trademark of Masimo Corporation.

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© 2022 Masimo Corporation
Rad-G este o marcă comercială înregistrată federal a companiei Masimo Corporation.
EU Authorized Representative for Masimo Corporation:
MDSS GmbH