
# LNCS® Patient Cables

## LNC Series




Images	2-3
<b>en</b> English	4-6
<b>fr</b> French	7-9
<b>de</b> German	10-12
<b>it</b> Italian	13-15
<b>es</b> Spanish	16-18
<b>sv</b> Swedish	19-21
<b>nl</b> Dutch	22-24
<b>da</b> Danish	25-27
<b>pt</b> Portuguese	28-30
<b>zh</b> Chinese	31-32
<b>ja</b> Japanese	33-35
<b>fi</b> Finnish	36-38
<b>no</b> Norwegian	39-41
<b>cs</b> Czech	42-44
<b>hu</b> Hungarian	45-47
<b>pl</b> Polish	48-50
<b>ro</b> Romanian	51-53
<b>sk</b> Slovak	54-56
<b>tr</b> Turkish	57-59
<b>el</b> Greek	60-62
<b>ru</b> Russian	63-65
<b>ar</b> Arabic	67-66
<b>fa</b> Farsi	69-68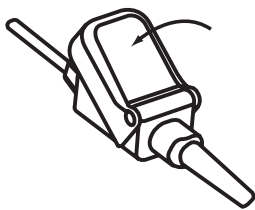


# LNCS<sup>®</sup> Patient Cables

## LNC Series

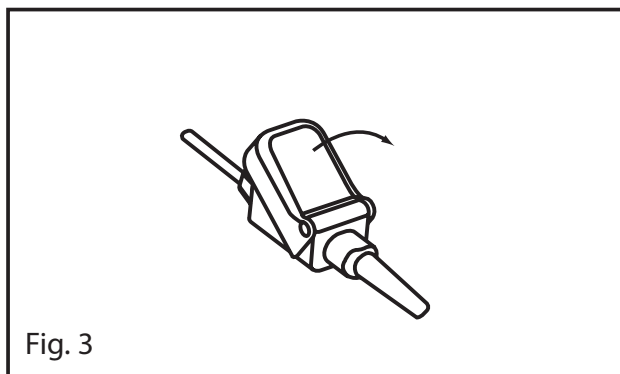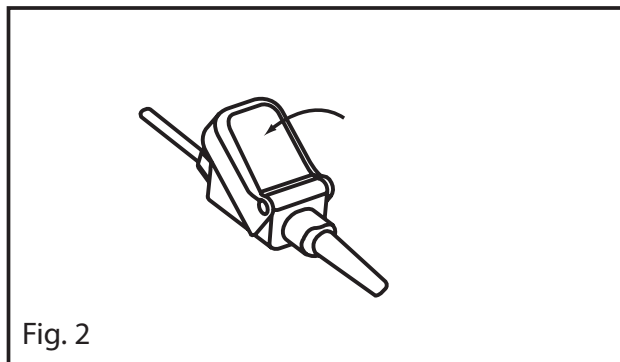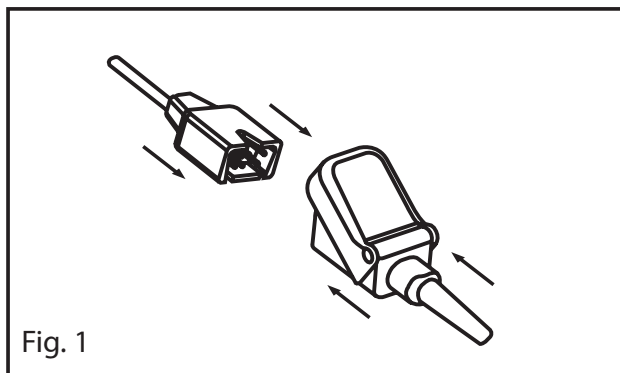# LNCS<sup>®</sup> Patient Cables

## LNC Series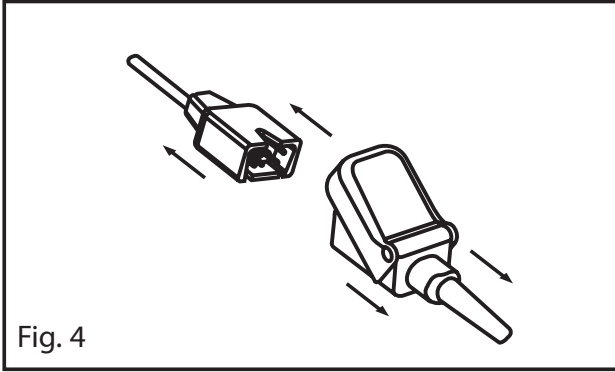

# LNCS® Patient Cables

## LNC Series

en

### DIRECTIONS FOR USE

Reusable

Not made with natural rubber latex

Non-Sterile

#### INDICATIONS

The LNC Patient Cable series have the same indications for use as the applicable sensors. Please see the compatible sensor's directions for use for indications for use and prescribed information.

#### DESCRIPTION

The LNC Patient Cable Series are used with Masimo LNCS® sensors. The LNC Patient Cable and Masimo LNCS Sensors have been validated with Masimo SET® Rad-5® Pulse Oximeter.

#### WARNINGS, CAUTIONS, AND NOTES

- Ensure the cable is physically intact, with no broken or frayed wires or damaged parts. Visually inspect the cable and discard if cracks or discoloration are found.
- Carefully route patient cable to reduce the possibility of patient entanglement or strangulation.
- Do not attempt to reprocess, recondition or recycle Masimo sensors or patient cables as these processes may damage the electrical components, potentially leading to patient harm.
- Masimo sensors and cables are designed for use with devices containing Masimo SET® oximetry or licensed to use Masimo sensors.
- All sensors and cables are designed for use with specific monitors. Verify the compatibility of the monitor, cable and sensor before use, otherwise degraded performance and/or patient injury can result.
- Failure to properly connect the sensor or the oximeter module to the cable will result in intermittent readings, inaccurate results, or no reading.
- To avoid damage to the cable, always hold it by the connector rather than the cable when connecting or disconnecting either end.
- Always refer to the oximeter module operator's manual for complete instructions or additional instructions.
- To prevent damage, do not soak or immerse the sensor or cable in any liquid solution. Do not attempt to sterilize the cable or sensor.
- **Caution:** Replace the cable when a replace cable message is displayed, or when a low SIQ message is consistently displayed while monitoring consecutive patients after completing the low SIQ troubleshooting steps identified in the monitoring device operator's manual.
- **Note:** The cable is provided with X-Cal® technology to minimize the risk of inaccurate readings and unanticipated loss of patient monitoring. The cable will provide up to 17,520 hours of patient monitoring time. Replace the cable when the patient monitoring time is exhausted. Some legacy monitoring devices do not distinguish between the cable and the sensor. In these devices, a replace sensor or similar message could pertain either to the sensor or the cable.

#### INSTRUCTIONS

##### A) Attach the LNC Patient Cable to the device

1. Orient the connector of the cable to mate with the connector on the patient cable connector of the Masimo SET instrument and insert.

##### B) Attaching the LNC Cable to the LNCS Sensor

1. Insert the sensor connector completely into the patient cable connector (Fig. 1).
2. Completely close the protective cover (Fig. 2).

##### C) Disconnecting the LNC Cable from the LNCS Sensor

1. Lift the protective cover to gain access to the sensor connector (Fig. 3).
2. Pull firmly on the sensor connector to remove from the patient cable (Fig. 4).

#### WARNING

Do not use the LNC-10 or LNC-14 Patient cables with the PXC2-30 Extension cable.

#### CLEANING

Clean the LNC Patient Cable by wiping it with a 70% isopropyl alcohol pad and allowing it to dry thoroughly.

---

## COMPATIBILITY

 LNC Patient Cables are for use only with devices containing Masimo SET® technology and with compatible sensors. Consult individual oximetry system manufacturers for compatibility of particular devices and sensor models. Each device manufacturer is responsible for determining whether their devices are compatible with each sensor and/or cable model.

For Compatibility Information Reference: [www.masimo.com](http://www.masimo.com)

---

## WARRANTY

Masimo warrants to the initial buyer only that this product, when used in accordance with the directions provided with the Products by Masimo, will be free of defects in materials and workmanship for a period of six (6) months. Single use products are warranted for single patient use only.

THE FOREGOING IS THE SOLE AND EXCLUSIVE WARRANTY APPLICABLE TO THE PRODUCTS SOLD BY MASIMO TO BUYER. MASIMO EXPRESSLY DISCLAIMS ALL OTHER ORAL, EXPRESS OR IMPLIED WARRANTIES, INCLUDING WITHOUT LIMITATION ANY WARRANTIES OF MERCHANTABILITY OR FITNESS FOR PARTICULAR PURPOSE. MASIMO'S SOLE OBLIGATION AND BUYER'S EXCLUSIVE REMEDY FOR BREACH OF ANY WARRANTY SHALL BE, AT MASIMO'S OPTION, TO REPAIR OR REPLACE THE PRODUCT.

---

## WARRANTY EXCLUSIONS

This warranty does not extend to any product that has been used in violation of the operating instructions supplied with the product, or has been subject to misuse, neglect, accident or externally created damage. This warranty does not extend to any product that has been connected to any unintended device or system, has been modified, or has been disassembled or reassembled. This warranty does not extend to sensors or patient cables that have been reprocessed, reconditioned or recycled.

IN NO EVENT SHALL MASIMO BE LIABLE TO BUYER OR ANY OTHER PERSON FOR ANY INCIDENTAL, INDIRECT, SPECIAL OR CONSEQUENTIAL DAMAGES (INCLUDING WITHOUT LIMITATION LOST PROFITS), EVEN IF ADVISED OF THE POSSIBILITY THEREOF. IN NO EVENT SHALL MASIMO'S LIABILITY ARISING FROM ANY PRODUCTS SOLD TO BUYER (UNDER A CONTRACT, WARRANTY, TORT OR OTHER CLAIM) EXCEED THE AMOUNT PAID BY BUYER FOR THE LOT OF PRODUCT(S) INVOLVED IN SUCH CLAIM. IN NO EVENT SHALL MASIMO BE LIABLE FOR ANY DAMAGES ASSOCIATED WITH A PRODUCT THAT HAS BEEN REPROCESSED, RECONDITIONED OR RECYCLED. THE LIMITATIONS IN THIS SECTION SHALL NOT BE DEEMED TO PRECLUDE ANY LIABILITY THAT, UNDER APPLICABLE PRODUCTS LIABILITY LAW, CANNOT LEGALLY BE PRECLUDED BY CONTRACT.

---

## NO IMPLIED LICENSE

Purchase or possession of this LNC Patient Cable does not carry any express or implied license to use this cable with any device that is not an authorized device or separately authorized to use LNC Patient Cables.

---

**CAUTION:** FEDERAL LAW (U.S.A.) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN.

For professional use. See instructions for use for full prescribing information, including indications, contraindications, warnings, precautions and adverse events.

If you encounter any serious incident with product, please notify the competent authority in your country and the manufacturer.

The following symbols may appear on the product or product labeling:

SYMBOL	DEFINITION	SYMBOL	DEFINITION	SYMBOL	DEFINITION
	Follow instructions for use		Separate collection for electrical and electronic equipment (WEEE).	<b>Rx ONLY</b>	<b>Caution:</b> Federal law (USA) restricts this device to sale by or on the order of a physician
	Consult instructions for use	<b>LOT</b>	Lot code	<b>CE</b>	Mark of conformity to European Medical Device Directive 93/42/EEC
	Manufacturer	<b>REF</b>	Catalogue number (model number)	<b>EC REP</b>	Authorized representative in the European community
	Date of manufacture YYYY-MM-DD	<b>####</b>	Masimo reference number		Body weight
	Caution		Storage humidity limitation		Storage temperature range
	Do not discard		Do not use if package is damaged and consult instructions for use		Keep dry
	Fragile, handle with care	<b>&gt;</b>	Greater than	<b>&lt;</b>	Less than
	Non-sterile		Not made with natural rubber latex		Atmospheric pressure limitation
<b>MD</b>	Medical device	<b>UDI</b>	Unique device identifier		
	Instructions/Directions for Use/Manuals are available in electronic format @ <a href="http://www.Masimo.com/TechDocs">http://www.Masimo.com/TechDocs</a> <b>Note: eIFU is not available in all countries.</b>				

Patents: <http://www.masimo.com/patents.htm>

Masimo, SET, , LNCS, X-Cal, and Rad-5 are federally registered trademarks of Masimo Corporation.

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© 2020 Masimo Corporation
Authorized representative for Masimo Corporation:
MDSS GmbH