
# LNCS® Extension Cables





## LNC-EXT Series

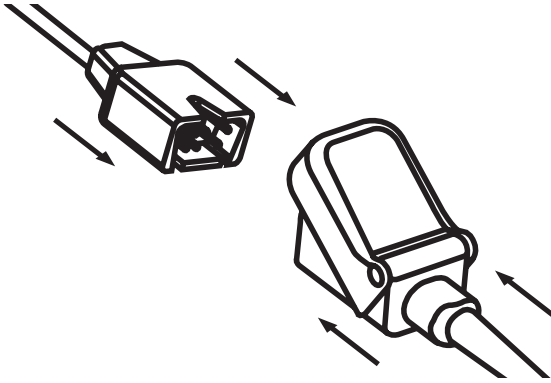Images	2
<b>en</b> English	3-5
<b>fr</b> French	6-8
<b>de</b> German	9-11
<b>it</b> Italian	12-14
<b>es</b> Spanish	15-17
<b>sv</b> Swedish	18-20
<b>nl</b> Dutch	21-23
<b>da</b> Danish	24-26
<b>pt</b> Portuguese	27-29
<b>zh</b> Chinese	30-32
<b>ja</b> Japanese	33-35

# LNCS® Extension Cables

## LNC-EXT Series

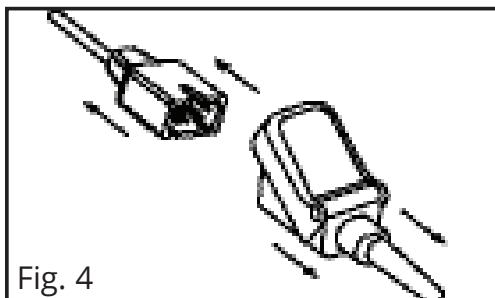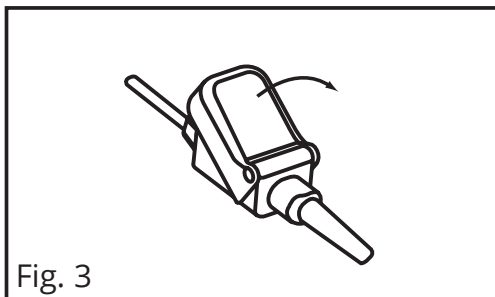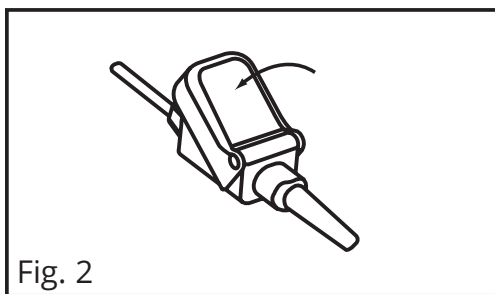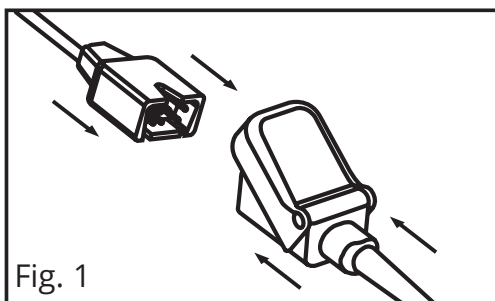## LNC-EXT Series

### DIRECTIONS FOR USE

Reusable

Not made with natural rubber latex

Non-Sterile

### INDICATIONS

The LNCS Series Extension Cable (LNC-EXT) is designed for compatibility with either Masimo LNCS Patient Cables used with Masimo or Masimo compatible pulse oximetry systems, or with Nellcor patient cables used with Nellcor or Nellcor compatible pulse oximetry systems.

The LNCS Extension Cable also provides a direct connect solution to Nellcor N180 pulse oximeters, or compatible devices utilizing Nellcor N200 pulse oximetry technology and having a DB9 connector interface at the host device.

### INSTRUCTIONS

#### I. For use with Masimo LNCS Patient Cables using Masimo SET pulse oximetry technology:

##### A) Verify your Patient Cable setup is compatible with the Patient Extension Cable.

Use Table 1 to determine if your setup qualifies:

Existing Setup	LNC-EXT Qualifies	Existing Setup	LNC-EXT Qualifies
LNC-04	yes	PXC30+LNC-04	yes
LNC-10	yes	PXC30+LNC-10	yes
LNC-14	yes	PXC30+LNC-14	yes
PXC16+LNC-04	yes	PXC50+LNC-04	yes
PXC16+LNC-10	yes	PXC50+LNC-10	no
PXC16+LNC-14	yes	PXC50+LNC-14	no

##### B) Attaching the LNC-EXT Cable to a LNCS Sensor or to a LNC Cable.

1. Insert the sensor cable connector completely into the LNC-EXT Cable (Fig. 1).
2. Completely close the protective cover (Fig. 2).

##### C) Disconnecting the LNCS Sensor Cable from the LNC Patient Extension Cable.

1. Lift the protective cover to gain access to the sensor connector (Fig. 3).
2. Pull firmly on the sensor connector to remove from the patient cable (Fig. 4).

#### II. For direct connection to Nellcor or Nellcor compatible devices:

##### A) Attaching the LNC-EXT Cable to a Nellcor or Nellcor compatible host device.

1. Insert the LNC-EXT sensor cable connector into the host device.

##### B) Attaching the LNC-EXT Cable to a LNCS Sensor.

1. Lift the protective cover on the connector (sensor end) of the LNC-EXT cable and insert the sensor cable connector completely into the LNC-EXT cable (Fig. 1).
2. Completely close the protective cover on the LNC-EXT cable (Fig. 2).

##### C) Disconnecting the LNCS Sensor from the LNC Extension Cable.

1. Lift the protective cover to gain access to the sensor connector (Fig. 3).
2. Pull firmly on the sensor connector to remove from the patient cable (Fig. 4).

---

## CLEANING

Clean the LNCS Series Patient Extension Cables by wiping it with a 70% isopropyl alcohol pad and allow it to dry.

---

## CAUTION

- Do not soak or immerse the cable in any liquid solution. Do not attempt to sterilize.
- 

## WARNINGS

- Ensure the cable is physically intact, with no broken or frayed wires or damaged parts. Visually inspect the cable and discard if cracks or discoloration are found.
  - Failure to properly connect the sensor or the oximeter module to the cable will result in intermittent readings, inaccurate results, or no reading.
  - To avoid damage to the cable, always hold it by the connector rather than the cable when connecting or disconnecting either end.
  - Always refer to the oximeter module operator's manual for complete instructions or additional instructions.
  - Carefully route the patient cable to reduce the possibility of patient entanglement or strangulation.
- 

## ENVIRONMENTAL

Operating Temperature	41°F to 104°F (5°C to 40°C)
Storage Temperature	-40°F to 158°F (-40°C to 70°C)
Relative Humidity	5% to 95% noncondensing

---

## WARRANTY

The manufacturer warrants to the initial purchaser that each new LNCS Series Patient Extension Cable will be free from defects in workmanship or materials for a period of six (6) months from the date of purchase. The manufacturer's sole obligation under this warranty is to replace any product that it deems to be covered under warranty with a replacement LNCS Series Patient Extension Cable.

---

## WARRANTY EXCLUSIONS

This warranty does not extend to any product that has been used in violation of the operating instructions supplied with the product, or has been subject to misuse, neglect, accident or externally created damage. This warranty does not extend to any product that has been connected to any unintended instrument or system, has been modified, or has been disassembled or reassembled.

THIS WARRANTY, TOGETHER WITH ANY OTHER EXPRESSED WRITTEN WARRANTY THAT MAY BE ISSUED BY THE MANUFACTURER IS THE SOLE AND EXCLUSIVE WARRANTY AS TO ITS PRODUCTS. THIS WARRANTY IS EXPRESSLY IN LIEU OF ANY ORAL OR IMPLIED WARRANTIES, INCLUDING WITHOUT LIMITATION ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. THE MANUFACTURER SHALL NOT BE LIABLE FOR ANY INCIDENTAL, SPECIAL, OR CONSEQUENTIAL LOSS, DAMAGE, OR EXPENSE DIRECTLY OR INDIRECTLY ARISING FROM THE USE, OR LOSS OF USE, OF ANY PRODUCTS.

---

## NO IMPLIED LICENSE

Purchase or possession of this LNCS Series Patient Extension Cable does not carry any express or implied license to use this cable with any device that is not an authorized device or separately authorized to use LNCS Series Patient Extension Cable.

**CAUTION:** FEDERAL LAW (U.S.A.) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN.

For professional use. See instructions for use for full prescribing information, including indications, contraindications, warnings, precautions and adverse events.

If you encounter any serious incident with product, please notify the competent authority in your country and the manufacturer.

The following symbols may appear on the product or product labeling:

SYMBOL	DEFINITION	SYMBOL	DEFINITION	SYMBOL	DEFINITION
	Follow instructions for use		Separate collection for electrical and electronic equipment (WEEE).	<b>Rx ONLY</b>	<b>Caution:</b> Federal law (USA) restricts this device to sale by or on the order of a physician
	Consult instructions for use	<b>LOT</b>	Lot code	<b>CE</b>	European Union Conformity Mark
	Manufacturer	<b>REF</b>	Catalogue number (model number)	<b>EC REP</b>	Authorized representative in the European community
	Date of manufacture YYYY-MM-DD	<b>###</b>	Masimo reference number		Body weight
	Caution		Storage humidity limitation		Storage temperature range
	Do not discard		Do not use if package is damaged and consult instructions for use		Keep dry
	Fragile, handle with care	<b>&gt;</b>	Greater than	<b>&lt;</b>	Less than
	Non-sterile		Not made with natural rubber latex		Atmospheric pressure limitation
<b>MD</b>	Medical device	<b>UDI</b>	Unique device identifier		Instructions/Directions for Use/Manuals are available in electronic format @ <a href="http://www.Masimo.com/TechDocs">http://www.Masimo.com/TechDocs</a> <b>Note: eIFU is not available in all countries.</b>
	Importer		Distributor		

Patents: <http://www.masimo.com/patents.htm>

Masimo, SET,  LNOP and LNCS are federally registered trademarks of Masimo Corporation.

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© 2021 Masimo Corporation
Nellcor is a registered trademark of Nellcor Puritan Bennett Incorporated.
EU Authorized Representative for Masimo Corporation:
MDSS GmbH