Masimo® rainbow® Tester

en	English	2-3
fr	French	4-5
de	German	6-7
it	Italian	8-9
es	Spanish	10-11
sv	Swedish	12-13
nl	Dutch	14-15
da	Danish	16-17
pt	Portuguese	18-19
zh	Chinese	20-21
ja	Japanese	22-23
fi	Finnish	24-25
no	Norwegian	26-27
cs	Czech	28-29
hu	Hungarian	30-31
pl	Polish	32-33
ro	Romanian	34-35
sk	Slovak	36-37
tr	Turkish	38-39
el	Greek	40-41
ru	Russian	42-43
ar	Arabic	45-44
fa	Farsi	47-46

Masimo® rainbow® Tester

en

DIRECTIONS FOR USE

Reusable

Not made with natural rubber latex

Non-sterile

PURPOSE

The Masimo® rainbow® Tester is used to verify the general functionality of Masimo devices or devices containing Masimo rainbow SET® Pulse CO-Oximetry. The Tester allows a connection to Masimo rainbow SET devices. Each device manufacturer is responsible for determining whether its devices are compatible for use with the Tester.

CONTRAINDICATIONS

The Tester is not to be used as a calibration tool for pulse CO-Oximeters.

WARNINGS

- The Tester should be free of visible defects. Never use a damaged product, or one with an open enclosure shell or exposed electrical contacts.
- Failure to properly connect the Tester to the device or patient cable will result in intermittent readings, inaccurate results or no reading.
- Functional Testers cannot be used to assess accuracy.

INSTRUCTIONS

A. Attaching the Tester to the Device or Patient Cable

Orient the Tester such that the connector mates with the connector port on the device or patient cable and insert firmly.

B. Taking a Test Measurement

1. Turn the device on.
2. A properly connected Tester will cause the device to display readings.
3. The resulting representative data points should be as follows:

SpO₂ = 81% ± 3%.

Pulse Rate = 61 bpm ± 1 bpm.

SpCO® = 7% ± 3%. *

SpMet® = 3.5% ± 3%. *

SpHb® = 14.0 g/dl ± 1 g/dl. *

*** If applicable**

NOTE: SpO₂ and SpMet will alarm at factory default alarm values.

NOTE: If the displayed values differ from the specified values, contact Masimo Technical Support. Contact information can be found at www.Masimo.com/globalservices

4. Remove the tester from the device or cable.

CLEANING

Clean the Tester by wiping with 70% isopropyl alcohol and allowing it to dry thoroughly.

CAUTION

Do not use petroleum-based or acetone solutions, or other harsh solvents for cleaning. These substances affect the devices materials and device failure can result.

WARRANTY

Masimo warrants to the initial buyer only that these products, when used in accordance with the directions provided with the Products by Masimo, will be free of defects in materials and workmanship or a period of six (6) months from the date of purchase. The manufacturer's sole obligation under this warranty is to replace any product that it deems to be covered under warranty with a replacement Tester.

WARRANTY EXCLUSIONS

This warranty does not extend to any product that has been subject to misuse, neglect or accident; that has been damaged by causes external to the product; or that has been used in violation of the operating instructions supplied with the product. The warranty does not extend to any product that has been connected to an unlicensed device system, modified accessories, or any unit that has been disassembled or reassembled by anyone but an authorized agent.

IN NO EVENT SHALL MASIMO BE LIABLE TO BUYER OR ANY OTHER PERSON FOR ANY INCIDENTAL, INDIRECT, SPECIAL OR CONSEQUENTIAL DAMAGES (INCLUDING WITHOUT LIMITATION LOST PROFITS), EVEN IF ADVISED OF THE POSSIBILITY THEREOF. IN NO EVENT SHALL MASIMO'S LIABILITY ARISING FROM ANY PRODUCTS SOLD TO BUYER (UNDER A CONTRACT, WARRANTY, TORT OR OTHER CLAIM) EXCEED THE AMOUNT PAID BY BUYER FOR THE LOT OF PRODUCT(S) INVOLVED IN SUCH CLAIM. IN NO EVENT SHALL MASIMO BE LIABLE FOR ANY DAMAGES ASSOCIATED WITH A PRODUCT THAT HAS BEEN REPROCESSED, RECONDITIONED OR RECYCLED. THE LIMITATIONS IN THIS SECTION SHALL NOT BE DEEMED TO PRECLUDE ANY LIABILITY THAT, UNDER APPLICABLE PRODUCTS LIABILITY LAW, CANNOT LEGALLY BE PRECLUDED BY CONTRACT.

NO IMPLIED LICENSE

PURCHASE OR POSSESSION OF THIS TESTER DOES NOT CARRY ANY EXPRESS OR IMPLIED LICENSE TO USE THIS DEVICE WITH ANY DEVICE THAT IS NOT AN AUTHORIZED DEVICE OR SEPARATELY AUTHORIZED TO USE THE TESTER.

CAUTION:

FEDERAL LAW (U.S.A.) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN.

For professional use. See instructions for use for full prescribing information, including indications, contraindications, warnings, precautions and adverse events.

The following symbols may appear on the product or product labeling:

SYMBOL	DEFINITION	SYMBOL	DEFINITION
	Caution		Authorized representative in the European community
	Follow instructions for use		Separate collection for electrical and electronic equipment (WEEE).
	Manufacturer		Lot code
	Date of Manufacture YYYY-MM-DD		Catalogue number (model number)
Rx ONLY	Caution: Federal law (USA) restricts this device to sale by or on the order of a physician.		Storage humidity limitation
	Non-Sterile		Storage temperature range
	Not made with natural rubber latex		Keep dry
	Mark of conformity to European Medical Device Directive 93/42/EEC		Fragile, handle with care
	Instructions/Directions for Use/Manuals are available in electronic format @ http://www.Masimo.com/TechDocs Note: eFU is not available for CE mark countries.		

Patents: <http://www.masimo.com/patents.htm>

Masimo, , SET, SpMet, SpHb, SpCO and Pulse CO-Oximeter are federally registered trademarks of Masimo Corporation.

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© 2016 Masimo Corporation
MDSS GmbH