M-LNCS® Sensor to LNC MAC Patient Cable Adapter

M-LNCS to LNC MAC Adapter









Images	2
en English	3-5
fr French	6-8
de German	9-11
it Italian	12-14
es Spanish	15-17
sv Swedish	18-20
nl Dutch	21-23
da Danish	24-26
pt Portuguese	27-29
zh Chinese	30-32
ja Japanese	33-35
fi Finnish	36-38
no Norwegian	39-41
cs Czech	42-44
hu Hungarian	45-47
pl Polish	48-50
ro Romanian	51-53
sk Slovak	54-56
tr Turkish	57-59
el Greek	60-62
ru Russian	63-65
ko Korean	66-68
ar Arabic	71-69

M-LNCS® Sensor to LNC MAC

Patient Cable Adapter

M-LNCS to LNC MAC Adapter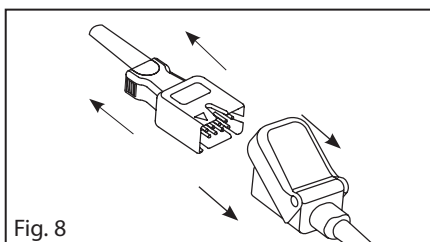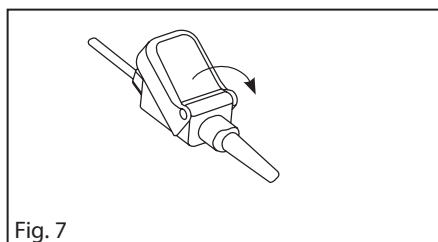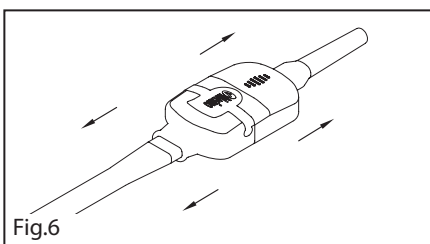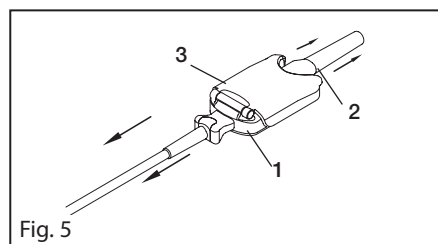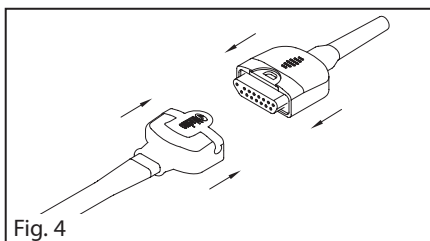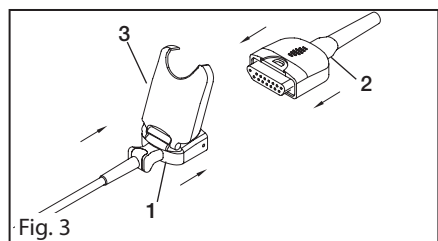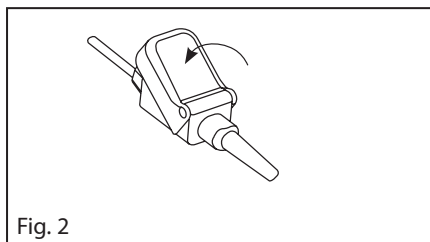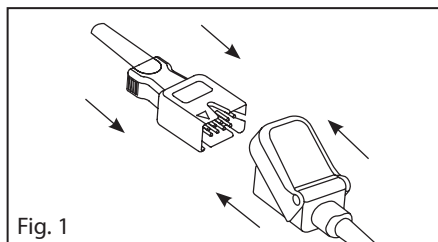

M-LNCS® Sensor to LNC MAC Patient Cable Adapter

en

M-LNCS to LNC MAC Adapter

DIRECTIONS FOR USE

Reusable

 Not made with natural rubber latex

 Non-sterile

INDICATIONS

The M-LNCS to LNC MAC Adapter Cable and Masimo M-LNCS sensors are indicated for the continuous noninvasive monitoring of arterial oxygen saturation (SpO₂) and pulse rate for adult, pediatric, infant and neonatal patients.

CONTRAINDICATIONS

The M-LNCS to LNC MAC Adapter cable is for use with Masimo M-LNCS sensor only.

DESCRIPTION

The M-LNCS to LNC MAC Adapter cable allows Masimo M-LNCS sensors to be used on devices with Nellcor and Nellcor compatible Oximeters. Masimo SET® Technology is not available when using this adapter cable.

The M-LNCS to LNC MAC Adapter Cable has been verified with Nellcor's N-200 oximeter.

WARNINGS

- Ensure the cable is physically intact, with no broken or frayed wires or damaged parts. Visually inspect the cable and discard if cracks or discoloration are found.
- Failure to properly connect the sensor or the oximeter module to the cable will result in intermittent readings, inaccurate results, or no reading.
- To avoid damage to the cable, always hold it by the connector rather than the cable when connecting or disconnecting either end.
- Always refer to the oximeter module operator's manual for complete instructions or additional instructions.
- Carefully route the patient cable to reduce the possibility of patient entanglement or strangulation.

INSTRUCTIONS

ATTACHING

A) Attaching the LNC MAC patient cable directly to the Instrument

Orient the connector of the cable to mate with the connector of the instrument.

B) Attaching the M-LNCS to LNC MAC Adapter to the LNC MAC patient Cable

1. Refer to **Fig. 1**. Insert the M-LNCS to LNC MAC Adapter completely into the LNC MAC patient cable connector.
2. Refer to **Fig. 2**. Completely close the protective cover of the LNC MAC patient cable.

C) Attaching the M-LNCS to LNC MAC Adapter to a M-LNCS Sensor

Reusable Sensor

1. Refer to **Fig. 3**. Orient the reusable sensor connector (1) to the M-LNCS to LNC MAC Adapter cable (2) as shown.
2. Insert the reusable sensor connector (1) completely into the M-LNCS to LNC MAC Adapter cable (2). Close the reusable sensor connector cover (3) over the M-LNCS to LNC MAC Adapter cable connector (2).

Adhesive Sensor

1. Refer to **Fig. 4**. Orient the adhesive sensor connector (1) to the M-LNCS to LNC MAC Adapter cable connector (2) as shown.
2. Insert the adhesive sensor connector (1) completely into the M-LNCS to LNC MAC Adapter cable connector (2) until it locks in place.

DISCONNECTING

Reusable Sensor

1. Refer to **Fig. 5**. While holding the sides of the M-LNCS to LNC MAC Adapter (2), lift the sensor cover (3) from the reusable sensor.
2. Disconnect the reusable sensor from the M-LNCS to LNC MAC Adapter.

Adhesive Sensor

1. Refer to **Fig. 6**. While holding the sides of the connectors, place thumb on the latch of the adhesive sensor connector and press.
2. Hold the latch down while pulling the connectors apart.

M-LNCS to LNC MAC Adapter from the LNC MAC patient cable

1. Refer to **Fig. 7**. Lift the protective cover of the LNC MAC patient cable to gain access to the connector.
2. Refer to **Fig. 8**. Pull firmly on the M-LNCS to LNC MAC Adapter cable connector to remove it from the LNC MAC patient cable.

CLEANING

Clean the M-LNCS to LNC MAC Adapter by wiping it with a 70% isopropyl alcohol pad and allow it to dry.

CAUTION

- Do not soak or immerse the cable in any liquid solution. Do not attempt to sterilize.

SPECIFICATIONS

The M-LNCS to LNC MAC Adapter Cable is for use with the following sensors:

Sensors	 Body Weight	Saturation Accuracy (70–100% SpO ₂)	Pulse Rate Accuracy (25–240 bpm)
		No Motion	No Motion
M-LNCS Adbx/Adbx3	> 30 kg	2%	3 bpm
M-LNCS Pdbx/Pdbx3	10–50 kg	2%	3 bpm
M-LNCS Inf/Inf-3	3–20 kg	2%	3 bpm
M-LNCS Neo/Neo-3	< 3 kg	3%	3 bpm
	> 40 kg	2%	3 bpm
M-LNCS NeoPt/NeoPt-3	< 1 kg	3%	3 bpm
M-LNCS DCI	> 30 kg	2%	3 bpm
M-LNCS DCIP	10–50 kg	2%	3 bpm
M-LNCS TC-I	> 30 kg	3.5%	3 bpm

NOTE: ARMS accuracy is a statistical calculation of the difference between device measurements and reference measurements. Approximately two-thirds of the device measurements fell within $\pm$ ARMS of the reference measurements in a controlled study. The saturation accuracy of the neonatal sensors were validated on adult volunteers and 1% was added to account for the properties of fetal hemoglobin.

ENVIRONMENTAL

Operating Temperature	41°F to 104°F (5°C to 40°C)
Storage Temperature	-40°F to 158°F (-40°C to 70°C)
Relative Humidity	Up to 95%, non-condensing

WARRANTY

Masimo warrants to the initial buyer only that this product, when used in accordance with the directions provided with the Products by Masimo, will be free of defects in materials and workmanship for a period of six (6) months. Single use products are warranted for single patient use only.

THE FOREGOING IS THE SOLE AND EXCLUSIVE WARRANTY APPLICABLE TO THE PRODUCTS SOLD BY MASIMO TO BUYER. MASIMO EXPRESSLY DISCLAIMS ALL OTHER ORAL, EXPRESS OR IMPLIED WARRANTIES, INCLUDING WITHOUT LIMITATION ANY WARRANTIES OF MERCHANTABILITY OR FITNESS FOR PARTICULAR PURPOSE. MASIMO’S SOLE OBLIGATION AND BUYER’S EXCLUSIVE REMEDY FOR BREACH OF ANY WARRANTY SHALL BE, AT MASIMO’S OPTION, TO REPAIR OR REPLACE THE PRODUCT.

WARRANTY EXCLUSIONS

This warranty does not extend to any product that has been used in violation of the operating instructions supplied with the product, or has been subject to misuse, neglect, accident or externally created damage. This warranty does not extend to any product that has been connected to any unintended instrument or system, has been modified, or has been disassembled or reassembled. This warranty does not extend to sensors or patient cables that have been reprocessed, reconditioned or recycled. IN NO EVENT SHALL MASIMO BE LIABLE TO BUYER OR ANY OTHER PERSON FOR ANY INCIDENTAL, INDIRECT, SPECIAL OR CONSEQUENTIAL DAMAGES (INCLUDING WITHOUT LIMITATION LOST PROFITS), EVEN IF ADVISED OF THE POSSIBILITY THEREOF. IN NO EVENT SHALL MASIMO’S LIABILITY ARISING FROM ANY PRODUCTS SOLD TO BUYER (UNDER A CONTRACT, WARRANTY, TORT OR OTHER CLAIM) EXCEED THE AMOUNT PAID BY BUYER FOR THE LOT OF PRODUCT(S) INVOLVED IN SUCH CLAIM. IN NO EVENT SHALL MASIMO BE LIABLE FOR ANY DAMAGES ASSOCIATED WITH A PRODUCT THAT HAS BEEN REPROCESSED, RECONDITIONED OR RECYCLED. THE LIMITATIONS IN THIS SECTION SHALL NOT BE DEEMED TO PRECLUDE ANY LIABILITY THAT, UNDER APPLICABLE PRODUCTS LIABILITY LAW, CANNOT LEGALLY BE PRECLUDED BY CONTRACT.

NO IMPLIED LICENSE

PURCHASE OR POSSESSION OF THIS M-LNCS TO LNC MAC ADAPTER CABLE DOES NOT CARRY ANY EXPRESS OR IMPLIED LICENSE TO USE THIS CABLE WITH ANY DEVICE THAT IS NOT AN AUTHORIZED DEVICE OR SEPARATELY AUTHORIZED TO USE LNC PATIENT CABLES.

CAUTION: FEDERAL LAW (U.S.A.) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN.

For professional use. See instructions for use for full prescribing information, including indications, contraindications, warnings, precautions and adverse events.

The following symbols may appear on the product or product labeling:

SYMBOL	DEFINITION	SYMBOL	DEFINITION
	Consult instructions for use		Separate collection for electrical and electronic equipment (WEEE).
	Follow instructions for use		Lot code
	Manufacturer		Catalogue number (model number)
	Date of manufacture YYYY-MM-DD		Masimo reference number
	Use by YYYY-MM-DD		Body weight
	Do not discard		Greater than
	Non-Sterile		Less than
	Not made with natural rubber latex		Storage humidity limitation
Rx ONLY	Caution: Federal law (USA) restricts this device to sale by or on the order of a physician		Storage temperature range
	Mark of conformity to European Medical Device Directive 93/42/EEC		Keep dry
	Authorized representative in the European community		Do not use if package is damaged
	Instructions/Directions for Use/Manuals are available in electronic format @ http://www.Masimo.com/TechDocs Note: eIFU is not available in all countries.		

Patents: <http://www.masimo.com/patents.htm>

Masimo, SET, M-LNCS and  are federally registered trademarks of Masimo Corporation.

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© 2019 Masimo Corporation
EU Authorized Representative for Masimo Corporation:
MDSS GmbH