14 Pin to 20 Pin Adapter Cable

DIRECTIONS FOR USE
Reusable  Not made with natural rubber latex  Non-sterile

INDICATIONS

The 14 pin to 20 pin adapter cable connects Masimo SET™ LNC series, M-LNC series or LNOP® PC series patient cables, or Masimo direct connect reusable sensors, or Masimo PCX2 series extension cables to Masimo rainbow® SET devices with a 20-pin instrument connector.

The 14 pin to 20 pin adapter cable is intended for use only with sensors and instruments or licensed devices containing Masimo rainbow SET technology.

CAUTION

Do not soak or immerse the connectors in any liquid solution. Do not attempt to sterilize.

WARNINGS

- Ensure the cable is physically intact, with no broken or frayed wires or damaged parts. Visually inspect the cable and discard if cracks or discoloration are found.
- To avoid damage to the patient or adapter cable, always hold it by the connector rather than the cable when connecting or disconnecting.
- Failure to properly connect the sensor or the oximeter module to the cable will result in intermittent readings, inaccurate results, or no reading.
- Masimo PCX extension cables are not compatible.
- rainbow RC series patient cables are not compatible.
- rainbow Pulse CO-Oximetry measurements are not available when used with the PCX2 Series extension cables.
- Always refer to the CO-Oximeter's *Operators Manual* for complete instructions or additional instructions.
- Carefully route cable and patient cable to reduce the possibility of patient entanglement or strangulation.
- Do not attempt to reprocess, recondition or recycle Masimo adapter cables as these processes may damage the electrical components, potentially leading to patient harm.

INSTRUCTIONS

- Attaching the 14 Pin to 20 Pin Adapter Cable to the Instrument
 - Orient the 20 pin red connector of the adapter cable to mate with the connector on the 20 pin instrument (Fig. 1).
 - Press firmly to ensure the adapter cable is fully engaged with the instrument. An audible click of the connection should be heard.
 - Gently tug on the adapter cable connector to ensure a secure connection. Do not pull on the cable.
- Attaching the 14 to 20 Pin Adapter Cable to the Patient Cable or PCX2 Extension Cable
 - Ensure that the adapter cable or PCX2 extension cable connector (Fig. 2).
 - Press firmly to ensure the patient cable or PCX2 extension cable is fully engaged with the adapter cable. An audible click of the connection should be heard.
 - Gently tug on the connectors to ensure a secure connection. Do not pull on the cable.

Disconnecting the 14 to 20 Pin Adapter Cable from the Patient Cable, PCX2 or Instrument

- Place thumb and index finger on the buttons on either side of the connector.
- Press firmly on the buttons and pull to disconnect.

CLEANING

Clean the 14 pin to 20 pin adapter cable with a mild detergent or 70% isopropyl alcohol pad and allow to dry.

ENVIRONMENTAL

Operating Temperature	41°F to 104°F (5°C to 40°C)
Storage Temperature	-40°F to 158°F (-40°C to 70°C)
Storage Humidity	86°F to 140°F (30°C to 60°C) maximum 95% humidity

WARRANTY

Masimo warrants to the initial buyer only that this product, when used in accordance with the directions provided with the Products by Masimo, will be free from defects in workmanship or materials for a period of six (6) months.

THE FOREGOING IS THE SOLE AND EXCLUSIVE WARRANTY APPLICABLE TO THE PRODUCTS SOLD BY MASIMO TO BUYER. MASIMO EXPRESSLY DISCLAIMS ALL OTHER ORAL, EXPRESS OR IMPLIED WARRANTIES, INCLUDING WITHOUT LIMITATION ANY WARRANTIES OF MERCHANTABILITY OR FITNESS FOR PARTICULAR PURPOSE. MASIMO'S SOLE OBLIGATION AND BUYER'S EXCLUSIVE REMEDY FOR BREACH OF ANY WARRANTY SHALL BE, AT MASIMO'S OPTION, TO REPAIR OR REPLACE THE PRODUCT.

WARRANTY EXCLUSIONS

This warranty does not extend to any product that has been used in violation of the operating instructions supplied with the product, or has been subject to misuse, neglect, accident or externally created damage. This warranty does not extend to any product that has been connected to any unintended instrument or system, has been modified, or has been disassembled or reassembled. This warranty does not extend to sensors or patient cables that have been reprocessed, reconditioned or recycled.

IN NO EVENT SHALL MASIMO BE LIABLE TO BUYER OR ANY OTHER PERSON FOR ANY INCIDENTAL, INDIRECT, SPECIAL OR CONSEQUENTIAL DAMAGES (INCLUDING WITHOUT LIMITATION LOST PROFITS), EVEN IF ADVISED OF THE POSSIBILITY THEREOF. IN NO EVENT SHALL MASIMO'S LIABILITY ARISING FROM ANY PRODUCTS SOLD TO BUYER (UNDER A CONTRACT, WARRANTY, TORT OR OTHER CLAIM) EXCEED THE AMOUNT PAID BY BUYER FOR THE LOT OF PRODUCTS INVOLVED IN SUCH CLAIM. IN NO EVENT SHALL MASIMO BE LIABLE FOR ANY DAMAGES ASSOCIATED A PRODUCT THAT HAS BEEN REPROCESSED, RECONDITIONED OR RECYCLED. THE LIMITATIONS IN THIS SECTION SHALL NOT BE DEEMED TO PRECLUDE ANY LIABILITY THAT, UNDER APPLICABLE PPRODUCTS LIABILITY LAW, CANNOT LEGALLY BE PRECLUDED BY CONTRACT.

INSTRUMENT COMPATIBILITY

This cable is intended for use only with instruments or licensed devices containing rainbow SET oximetry technology. Consult individual manufacturer for compatibility with specific instruments and sensors. Use of this cable with other instruments may result in no or improper performance.

NO IMPLIED LICENSE

Purchase or possession of this adapter cable does not carry any express or implied license to use this cable with any device that is not an authorized device or separately authorized to use a 14 pin to 20 pin adapter cable.

CAUTION: FEDERAL LAW (U.S.A.) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN.

For professional use. See instructions for full prescribing information, including indications, contraindications, warnings, precautions and adverse events.

The following symbols may appear on the product or product labeling:

SYMBOL	DEFINITION	SYMBOL	DEFINITION
	Caution		Storage temperature range
Rx ONLY	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.		Storage humidity limitation
	Mark of conformity to European medical device directive 93/42/EEC		Do not use if package is damaged
	Manufacturer		Separate collection for electrical and electronic equipment (WEEE).
	Date of manufacture YYYY-MM-DD		Do not discard
ECIREP	Authorized representative in the European community		Keep dry
REF	Catalogue number (model number)		Not made with natural rubber latex
	Masimo reference number		Atmospheric pressure limitation
LOT	Lot code		Fragile, handle with care
	Non-Sterile		Follow Instructions for Use (blue background)

Patents: <http://www.masimo.com/patents.htm>

Masimo, SET,  LNOP, LNCs and rainbow are federally registered trademarks of Masimo Corporation.

M-LNCs is a trademark of Masimo Corporation.

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