PXC2 Series








Mountable Extension Cables

Images	2
en English	3-5
fr French	6-8
de German	9-11
it Italian	12-14
es Spanish	15-17
sv Swedish	18-20
nl Dutch	21-23
da Danish	24-26
pt Portuguese	27-29
zh Chinese	30-32
ja Japanese	33-35
fi Finnish	36-38
no Norwegian	39-41
cs Czech	42-44
hu Hungarian	45-47
pl Polish	48-50
ro Romanian	51-53
sk Slovak	54-56
tr Turkish	57-59
el Greek	60-62
ru Russian	63-65
ko Korean	66-68
ar Arabic	71-69

PXC2 Series

Mountable Extension Cables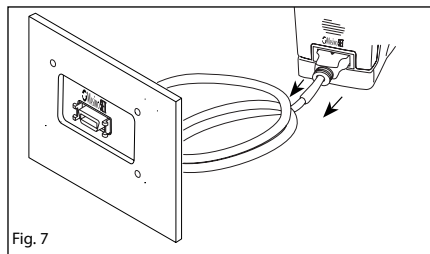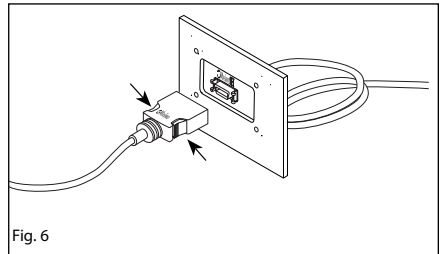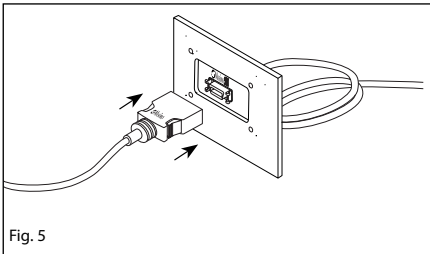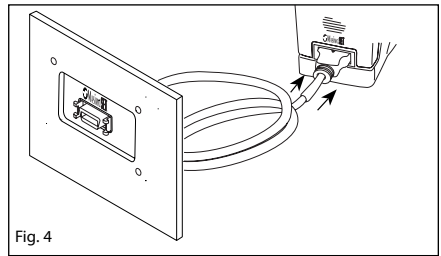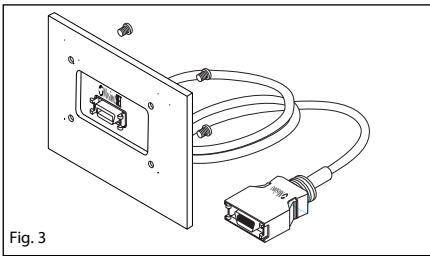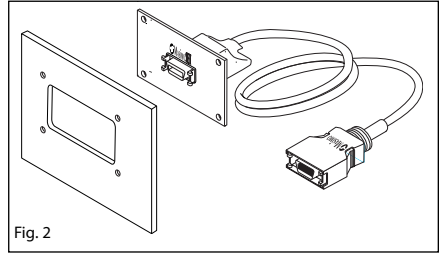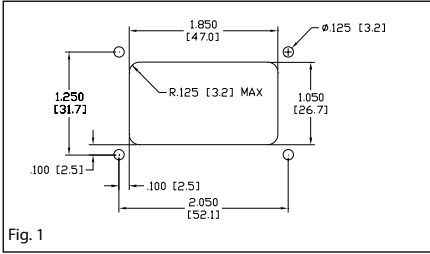

PXC2 Series

Mountable Extension Cables

DIRECTIONS FOR USE

Reusable

Not made with natural rubber latex

Non-sterile

Prior to using this cable, the user should read and understand the Operator's Manual for the Device or Monitor and this Directions for Use.

INDICATIONS

The PXC2 Mountable Extension Cable is intended for use in systems incorporating Masimo® SET® Pulse Oximeter technology which are intended for continuous noninvasive monitoring of functional oxygen saturation of arterial hemoglobin (SpO₂) and pulse rate (measured by a SpO₂ sensor) for adult, pediatric, and neonatal patients. The cable may be used in either a hospital or a mobile environment.

DESCRIPTION

The PXC2 extension cable is designed for installation into the wall or ceiling, allowing connection to Masimo SET LNC series, M-LNC series or LNOP® PC series patient cables, RD SET™ MD14 series or Masimo direct connect reusable sensors. The PXC2 extension cable is intended for use with Masimo sensors and oximeters licensed to use Masimo SET® technology.

WARNING: Masimo sensors and cables are designed for use with devices containing SET® oximetry or licensed to use Masimo sensors.

WARNINGS, CAUTIONS AND NOTES

- Ensure the cable is physically intact, with no broken or frayed wires or damaged parts. Visually inspect the cable and discard if cracks or discoloration are found.
- The PXC2 extension cable is designed to be connected directly to a compatible cable, sensor or 14 pin to 20 pin adapter cable and the oximeter module.
- Never connect the PXC2 extension cable to another PXC2 extension cable.
- To avoid damage to the patient or adapter cable, always hold it by the connector rather than the cable when connecting or disconnecting.
- Failure to properly attach the connector to the oximeter module and/or the PXC2 extension cable connector will result in intermittent readings, inaccurate results, or no reading.
- rainbow CO-Oximetry measurements are not available when the PXC2 extension cable is used with the 14 pin to 20 pin adapter cable on rainbow SET™ technology.
- Always refer to the oximeter module Operator's Manual for complete instructions or additional instructions.
- Carefully route cable and patient cable to reduce the possibility of patient entanglement or strangulation.
- Do not attempt to reprocess, recondition or recycle Masimo patient cables as these processes may damage the electrical components, potentially leading to patient harm.

MOUNTING INSTALLATION INSTRUCTIONS

1. Refer to **Fig. 1**. Punch or machine the panel the extension cables will be mounted into (following the dimensions given in **Fig. 1**). These dimensions are required to meet the minimum creepage distance requirements of IEC/ EN60601-1 section 57.10 d, table XVI.
2. Refer to **Fig. 2**. Place the panel mount connector behind the panel, with the Masimo logo in the top position.
3. Refer to **Fig. 3**. Attach using four (4) #4-40 or M3 screws (shown) and nuts (not shown). Use a thread locking compound and tighten the screws firmly.

NOTE: Drawing dimensions are in inches. Dimensions in [brackets] are in millimeters.

CONNECTION INSTRUCTIONS

Connecting the PXC2 to a Masimo SET 14-Pin Device or rainbow SET 20-Pin Device

Masimo SET 14-Pin Device

1. Orient the connector of the PXC2 cable to mate with the connector on the patient cable connector of the Masimo SET device.

rainbow SET 20-Pin Device, Using a 14-Pin to 20-Pin Adapter Cable

1. Orient the 20 pin red connector of the 14-pin to 20-pin adapter cable to mate with the connector on the patient cable connector of the Masimo rainbow SET device.
2. Orient the connector of the PXC2 cable to mate with the 14-pin connector of the 14-pin to 20-pin adapter cable.

Connecting a Patient Cable or Direct Connect Sensor to the PXC2 Cable

1. Refer to **Fig. 5**. Orient the connector of the patient cable or Direct Connect sensor to mate with the connector on the PXC2 panel mount.

Disconnecting a Patient Cable or Direct Connect Sensor to the PXC2 Cable

1. Refer to **Fig. 6**. Place the thumb and index finger on the buttons on either side of the connector.
2. Refer to **Fig. 7**. Press firmly on the buttons and pull to disconnect.

NOTE: Refer to specific sensor/cable Directions for Use for more detailed information.

CLEANING

Clean the PXC2 extension cable with a mild detergent or 70% isopropyl alcohol pad and allow to dry.

CAUTION: Do not soak or immerse the connectors in any liquid solution. Do not attempt to sterilize.

ENVIRONMENTAL

Operating Temperature & Humidity	41°F to 104°F (5°C to 40°C) 10-95% humidity (non-condensing)
Storage Temperature & Humidity	-4°F to 140°F (-20°C to +60°C), up to 95% humidity (non-condensing)

COMPATIBILITY

 This cable is intended for use only with devices or licensed devices containing Masimo SET oximetry technology. This cable can be used with devices or licensed devices containing rainbow SET oximetry technology using the Masimo SET 14 Pin to 20 Pin Adapter Cable. Pulse CO-Oximetry measurements are not available when the PXC2 Extension Cable is used with the 14 Pin to 20 Pin Adapter Cable. Use of this cable with other devices may result in no or improper performance.

 This cable is intended for use only with devices or licensed devices containing Masimo SET oximetry technology. This cable can be used with devices or licensed devices containing rainbow SET oximetry technology using the Masimo SET 14 Pin to 20 Pin Adapter Cable. Pulse CO-Oximetry measurements are not available when the PXC2 Extension Cable is used with the 14 Pin to 20 Pin Adapter Cable. Use of this cable with other devices may result in no or improper performance.

WARRANTY

Masimo warrants to the initial buyer only that this product, when used in accordance with the directions provided with the Products by Masimo, will be free of defects in materials and workmanship for a period of six (6) months. Single use products are warranted for single patient use only.

THE FOREGOING IS THE SOLE AND EXCLUSIVE WARRANTY APPLICABLE TO THE PRODUCTS SOLD BY MASIMO TO BUYER. MASIMO EXPRESSLY DISCLAIMS ALL OTHER ORAL, EXPRESS OR IMPLIED WARRANTIES, INCLUDING WITHOUT LIMITATION ANY WARRANTIES OF MERCHANTABILITY OR FITNESS FOR PARTICULAR PURPOSE. MASIMO'S SOLE OBLIGATION AND BUYER'S EXCLUSIVE REMEDY FOR BREACH OF ANY WARRANTY SHALL BE, AT MASIMO'S OPTION, TO REPAIR OR REPLACE THE PRODUCT.

WARRANTY EXCLUSIONS

This warranty does not extend to any product that has been used in violation of the operating instructions supplied with the product, or has been subject to misuse, neglect, accident or externally created damage. This warranty does not extend to any product that has been connected to any unintended device or system, has been modified, or has been disassembled or reassembled. This warranty does not extend to sensors or patient cables that have been reprocessed, reconditioned or recycled.

IN NO EVENT SHALL MASIMO BE LIABLE TO BUYER OR ANY OTHER PERSON FOR ANY INCIDENTAL, INDIRECT, SPECIAL OR CONSEQUENTIAL DAMAGES (INCLUDING WITHOUT LIMITATION LOST PROFITS), EVEN IF ADVISED OF THE POSSIBILITY THEREOF. IN NO EVENT SHALL MASIMO'S LIABILITY ARISING FROM ANY PRODUCTS SOLD TO BUYER (UNDER A CONTRACT, WARRANTY, TORT OR OTHER CLAIM) EXCEED THE AMOUNT PAID BY BUYER FOR THE LOT OF PRODUCT(S) INVOLVED IN SUCH CLAIM. IN NO EVENT SHALL MASIMO BE LIABLE FOR ANY DAMAGES ASSOCIATED A PRODUCT THAT HAS BEEN REPROCESSED, RECONDITIONED OR RECYCLED. THE LIMITATIONS IN THIS SECTION SHALL NOT BE DEEMED TO PRECLUDE ANY LIABILITY THAT, UNDER APPLICABLE PRODUCTS LIABILITY LAW, CANNOT LEGALLY BE PRECLUDED BY CONTRACT.

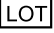
NO IMPLIED LICENSE

Purchase or possession of this extension cable does not carry any express or implied license to use this cable with any device that is not an authorized device or separately authorized to use PXC2 Series extension cables.

CAUTION: FEDERAL LAW (U.S.A.) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN.

For professional use. See instructions for use for full prescribing information, including indications, contraindications, warnings, precautions and adverse events.

The following symbols may appear on the product or product labeling:

SYMBOL	DEFINITION	SYMBOL	DEFINITION	SYMBOL	DEFINITION
	Follow instructions for use		Separate collection for electrical and electronic equipment (WEEE).	Rx ONLY	Caution: Federal law (USA) restricts this device to sale by or on the order of a physician
	Consult instructions for use		Lot code		Mark of conformity to European Medical Device Directive 93/42/EEC
	Manufacturer		Catalogue number (model number)		Authorized representative in the European community
	Date of manufacture YYYY-MM-DD		Masimo reference number		Body weight
	Caution		Storage humidity limitation		Storage temperature range
	Do not discard		Do not use if package is damaged		Keep dry
	Fragile, handle with care		Greater than		Less than
	Non-sterile		Not made with natural rubber latex		Atmospheric pressure limitation
	Instructions/Directions for Use/Manuals are available in electronic format @ http://www.Masimo.com/TechDocs Note: eIFU is not available for CE mark countries.				

Patents: <http://www.masimo.com/patents.htm>

Masimo, SET, , LNOP and rainbow are federally registered trademarks of Masimo Corporation.

rainbow SET and RD SET are trademarks of Masimo Corporation.

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© 2018 Masimo Corporation
EU Authorized Representative for Masimo Corporation:
MDSS GmbH