# rainbow® Extension Cable





## RC-EXT

Images	2
<b>en</b> English	3-5
<b>fr</b> French	6-8
<b>de</b> German	9-11
<b>it</b> Italian	12-14
<b>es</b> Spanish	15-17
<b>sv</b> Swedish	18-20
<b>nl</b> Dutch	21-23
<b>da</b> Danish	24-26
<b>pt</b> Portuguese	27-29
<b>zh</b> Chinese	30-32
<b>ja</b> Japanese	33-35
<b>fi</b> Finnish	36-38
<b>no</b> Norwegian	39-41
<b>cs</b> Czech	42-44
<b>hu</b> Hungarian	45-47
<b>pl</b> Polish	48-50
<b>ro</b> Romanian	51-53
<b>sk</b> Slovak	54-56
<b>tr</b> Turkish	57-59
<b>el</b> Greek	60-62
<b>ru</b> Russian	63-65
<b>ko</b> Korean	66-68
<b>ar</b> Arabic	71-69

# rainbow® Extension Cable

## RC-EXT

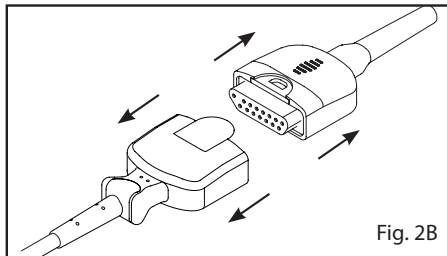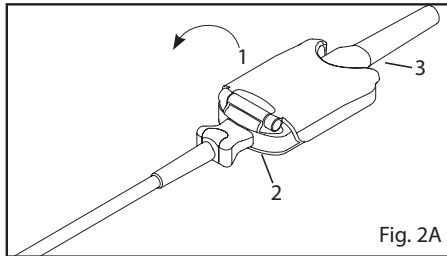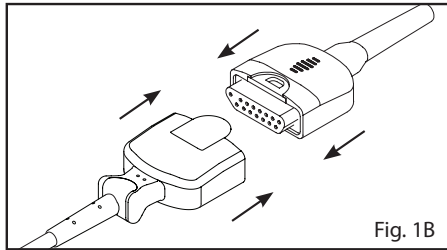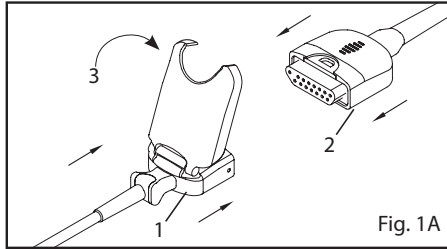## RC-EXT

### DIRECTIONS FOR USE

Reusable

Not made with natural rubber latex

Non-Sterile

Prior to using this Cable the user should read and understand the Operator's Manual for the Device or Monitor, the Directions for Use for the Sensor and Cable and this Directions for Use.

### DESCRIPTION

The rainbow® Extension Cable, RC-EXT, is designed for use with Masimo M-LNC or RC patient cables on Masimo or Masimo compatible pulse oximetry systems. When used with M-LNC patient cables, the RC-EXT is able to provide Masimo SET® (SpO<sub>2</sub> and PR) parameters. When used with RC patient cables, the RC-EXT is able to provide Masimo rainbow SET™ (SpO<sub>2</sub>, PR, SpCO®, SpMet®, SpHb®) parameters.

### WARNINGS

- Always refer to the oximeter module operator's manual for complete instructions or additional instructions.
- Ensure the cable is physically intact, with no broken or frayed wires or damaged parts. Visually inspect the cable and discard if cracks or discoloration are found.
- All sensors and cables are designed for use with specific monitors. Verify the compatibility of the monitor, cable and sensor before use, otherwise degraded performance and/or patient injury can result.
- Carefully route patient cable to reduce the possibility of patient entanglement or strangulation.
- Failure to properly connect the sensor or the oximeter module to the cable will result in intermittent readings, inaccurate results, or no reading.
- To avoid damage to the cable, always hold it by the connector rather than the cable when connecting or disconnecting either end.
- To prevent damage, do not soak or immerse the sensor or cable in any liquid solution. Do not attempt to sterilize the cable or sensor.
- Do not attempt to reprocess, recondition or recycle Masimo sensors or patient cables as these processes may damage the electrical components, potentially leading to patient harm.

### INSTRUCTIONS

**NOTE:** The maximum length for the sensor, cable and extension cable combination is 15 feet.

#### Connecting

##### to the Patient Cable

Refer to **Fig. 1A**. RC-EXT with protective cover: Properly orient the RC-EXT connector (1), insert the patient cable connector (2) to the RC-EXT connector and completely close the protective cover of the RC-EXT (3).

Refer to **Fig. 1B**. RC-EXT without protective cover: Properly orient the RC-EXT connector and insert the patient cable connector to the RC-EXT connector.

## **Disconnecting**

### **from the Patient Cable**

Refer to **Fig. 2A**. RC-EXT with protective cover: Lift the protective cover (1) to gain access to the RC-EXT connector (2) and pull firmly on the RC-EXT connector (2) to disconnect it (3).

Refer to **Fig. 2B**. RC-EXT without protective cover: Pull firmly on the RC-EXT connector to disconnect it.

**NOTE:** To avoid damage, pull on the RC-EXT connector, not the cable.

---

## **CLEANING**

Clean the RC-EXT cable by wiping it with a 70% isopropyl alcohol pad and allow it to dry.

---

## **CAUTION**

Do not soak or immerse the cable in any liquid solution. Do not attempt to sterilize.

---

## **ENVIRONMENTAL**

Operating Temperature	41°F to 104°F (5°C to 40°C)
Storage Temperature	-40°F to 158°F (-40°C to 70°C)
Relative Humidity	15% to 95% noncondensing

---

## **WARRANTY**

Masimo warrants to the initial buyer only that this product, when used in accordance with the directions provided with the Products by Masimo, will be free of defects in materials and workmanship for a period of six (6) months. Single use products are warranted for single patient use only.

THE FOREGOING IS THE SOLE AND EXCLUSIVE WARRANTY APPLICABLE TO THE PRODUCTS SOLD BY MASIMO TO BUYER. MASIMO EXPRESSLY DISCLAIMS ALL OTHER ORAL, EXPRESS OR IMPLIED WARRANTIES, INCLUDING WITHOUT LIMITATION ANY WARRANTIES OF MERCHANTABILITY OR FITNESS FOR PARTICULAR PURPOSE. MASIMO'S SOLE OBLIGATION AND BUYER'S EXCLUSIVE REMEDY FOR BREACH OF ANY WARRANTY SHALL BE, AT MASIMO'S OPTION, TO REPAIR OR REPLACE THE PRODUCT.

---

## **WARRANTY EXCLUSIONS**

This warranty does not extend to any product that has been used in violation of the operating instructions supplied with the product, or has been subject to misuse, neglect, accident or externally created damage. This warranty does not extend to any product that has been connected to any unintended instrument or system, has been modified, or has been disassembled or reassembled. This warranty does not extend to sensors or patient cables that have been reprocessed, reconditioned or recycled.

IN NO EVENT SHALL MASIMO BE LIABLE TO BUYER OR ANY OTHER PERSON FOR ANY INCIDENTAL, INDIRECT, SPECIAL OR CONSEQUENTIAL DAMAGES (INCLUDING WITHOUT LIMITATION LOST PROFITS), EVEN IF ADVISED OF THE POSSIBILITY THEREOF. IN NO EVENT SHALL MASIMO'S LIABILITY ARISING FROM ANY PRODUCTS SOLD TO BUYER (UNDER A CONTRACT, WARRANTY, TORT OR OTHER CLAIM) EXCEED THE AMOUNT PAID BY BUYER FOR THE LOT OF PRODUCT(S) INVOLVED IN SUCH CLAIM. IN NO EVENT SHALL MASIMO BE LIABLE FOR ANY DAMAGES ASSOCIATED A PRODUCT THAT HAS BEEN REPROCESSED, RECONDITIONED OR RECYCLED. THE LIMITATIONS IN THIS SECTION SHALL NOT BE DEEMED TO PRECLUDE ANY LIABILITY THAT, UNDER APPLICABLE PRODUCTS LIABILITY LAW, CANNOT LEGALLY BE PRECLUDED BY CONTRACT.

## NO IMPLIED LICENSE

Purchase or possession of these extension cables does not carry any express or implied license to use this cable with any device that is not an authorized device or separately authorized to use rainbow extension cables.

**CAUTION: FEDERAL LAW (U.S.A.) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN.**

For professional use. See instructions for use for full prescribing information, including indications, contraindications, warnings, precautions and adverse events.

The following symbols may appear on the product or product labeling:

SYMBOL	DEFINITION	SYMBOL	DEFINITION	SYMBOL	DEFINITION
	Follow instructions for use		Separate collection for electrical and electronic equipment (WEEE).	<b>Rx ONLY</b>	Federal law (USA) restricts this device to sale by or on the order of a physician
	Consult instructions for use	<b>LOT</b>	Lot code	<b>CE</b>	Mark of conformity to European Medical Device Directive 93/42/EEC
	Manufacturer	<b>REF</b>	Catalogue number (model number)	<b>EC REP</b>	Authorized representative in the European community
	Date of manufacture YYYY-MM-DD		Masimo reference number		Body weight
	Use-by YYYY-MM-DD		Storage humidity limitation		Storage temperature range
	Do not discard		Do not use if package is damaged		Keep dry
	Pediatric patient	<b>&gt;</b>	Greater than	<b>&lt;</b>	Less than
	Non-Sterile		Not made with natural rubber latex		Atmospheric pressure limitation
	Instructions/Directions for Use/Manuals are available in electronic format @ <a href="http://www.masimo.com/TechDocs">http://www.masimo.com/TechDocs</a> <b>Note: eFU is not available for CE mark countries.</b>				

Patents: <http://www.masimo.com/patents.htm>

rainbow SET is a trademark of Masimo Corporation.

Masimo, , SpCO, SpMet, SpHb and SET are federally registered trademarks of Masimo Corporation.

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© 2018 Masimo Corporation
EU Authorized Representative for Masimo Corporation:
MDSS GmbH