Masimo® Iris™ Adapter

Images	2
en English	3-5
fr French	6-8
de German	9-12
it Italian	13-15
es Spanish	16-18
sv Swedish	19-21
nl Dutch	22-24
da Danish	25-27
pt Portuguese	28-30
zh Chinese	31-33
ja Japanese	34-36
fi Finnish	37-39
no Norwegian	40-42
cs Czech	43-45
hu Hungarian	46-48
pl Polish	49-51
ro Romanian	52-54
sk Slovak	55-57
tr Turkish	58-60
el Greek	61-64
ru Russian	65-68
ko Korean	69-71
ar Arabic	74-72

Masimo® Iris™ Adapter

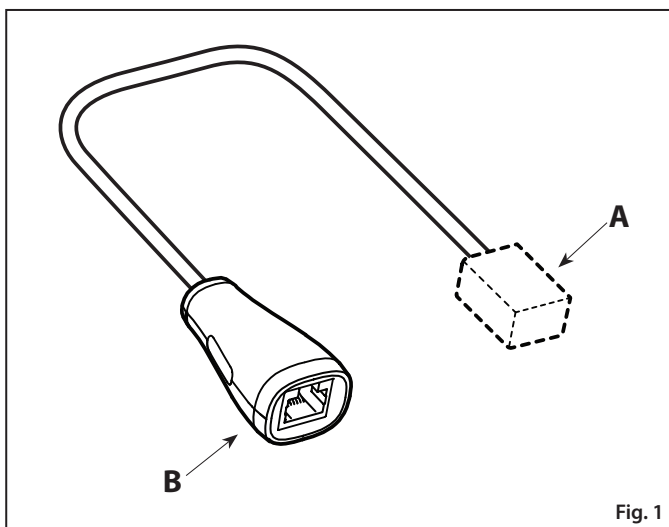
Fig. 1

Masimo[®]

Iris[™] Adapter

en

DIRECTIONS FOR USE

Reusable

Not made with natural rubber latex

Non-Sterile

Prior to using this Adapter, the user should read and understand the Operator's Manual for the device, monitor, cables, software/systems and this Directions for Use.

INTENDED USE

The Iris Adapter (Adapter) is intended to be used as a communication means between 3rd party medical devices and Masimo Gateway via Iris connectivity ports.

DESCRIPTION

The Iris Adapter is a Masimo proprietary adapter, wired or wireless, which establishes a data communication between 3rd party medical devices (POCs) with Masimo Gateway, via Iris connectivity ports.

The Iris Adapter allows a variety of standalone devices (such as: IV pumps, ventilators, beds, and other patient monitors) to connect to Root[®], where the "3rd party" device data can be passed through Root to Masimo systems, such as Patient SafetyNet[™] or Iris Gateway[™] and send the data to other systems.

For operation, the Iris Adapter requires Root and the Iris Gateway or Patient SafetyNet Supplemental Alarm System Series 5.0.0.0 (or higher) and compatible 3rd party device.

WARNINGS, CAUTIONS AND NOTES

- For complete safety information, consult the operator's manual for Root, and Iris Gateway or the Masimo Patient SafetyNet Supplemental Alarm System Series 5.0.0.0 (or higher).
- **Warning:** Disposal of product - Comply with local laws in the disposal of the device and/or its accessories.
- **Warning:** Do not place containers with liquids on or near the Iris Adapter. Liquids spilled on the Iris Adapter may cause it to perform inaccurately or fail.
- **Caution:** To prevent damage, do not soak or submerge the Iris Adapter in any cleaning solution or attempt to sterilize by autoclave, irradiation, steam, gas, ethylene oxide, or any other method. This will seriously damage the Iris Adapter.
- **Note:** Check the compatibility of the Iris adapter with the 3rd party device before use. The Iris adapter is designed to work properly only with compatible 3rd party devices.

INSTRUCTIONS

A) Using the Iris Adapter

To connect a standalone device using Iris Adapter

1. Connect the "standalone" or "3rd party" device to to the Iris Adapter. See **Fig. 1 (A)**.
2. Use an Ethernet cable to connect end **Fig. 1 (B)** of the Iris Adapter to any of the Four (4) Iris connectivity ports on Root.

To confirm that the standalone device has successfully connected, see:

1. The the "standalone" or "3rd party" device's User Manual.
2. The Operator's Manual for Root.
3. The Operator's Manual for Patient SafetyNet Supplemental Alarm System Series 5.0.0.0 or higher or Iris Gateway.

CLEANING

Wipe down the outer surface using any of the following solvents: Cidex Plus (3.4% glutaraldehyde), 10% bleach solution, or 70% isopropyl alcohol solution. Do not allow liquids to enter the interior.

CAUTION:

- Do not immerse the connector in any liquid solution.
- Do not sterilize by irradiation, steam, autoclave, or ethylene oxide.
- Do not clean with chemicals not approved above.

ENVIRONMENTAL

Dimensions	1.0" H x 2.0" W x 3.0" L (2.54 cm x 5.08 cm x 7.62 cm)
Operating Temperature	32°F to 158°F (0°C to 70°C)

WARRANTY

Masimo warrants to the initial Purchaser for a period of one (1) year from the date of purchase that: each new Product and the Software media as delivered are free from defects in workmanship or materials.

To request a replacement under warranty, Purchaser must contact Masimo for a returned goods authorization. If Masimo determines that a Product must be replaced under warranty, it will be replaced and the cost of shipment covered. All other shipping costs shall be the responsibility of Purchaser.

Masimo's sole obligation under this warranty is to replace any product that it deems to be covered under warranty with a replacement Iris Adapter.

WARRANTY EXCLUSIONS

The warranty does not extend to, and Masimo is not responsible for, repair, replacement, or maintenance needed because of: a) modification of the Product or Software without Masimo's written authorization; b) supplies, devices or electrical work external to the Product or not manufactured by Masimo; c) disassembly or reassembly of the Product by anyone other than an authorized Masimo agent; d) use of the Product with Sensors or other accessories other than those manufactured and distributed by Masimo; e) use of the Product and Software in ways or in environments for which they are not labeled; and f) neglect, misuse, improper operation, accident, fire, water, vandalism, weather, war, or any act of God.

This warranty does not extend to any product that has been used in violation of the operating instructions supplied with the product. This warranty does not extend to any Product that has been reprocessed, reconditioned or recycled.

This warranty also does not apply to any Products provided to Purchaser for testing or demonstration purposes, any temporary Products Modules or any Products for which Seller does not otherwise receive a usage or purchase fee; all such Products are provided AS-IS without warranty.

This warranty, together with any other express written warranty that may be issued by Masimo is the sole and exclusive warranty as to the Product and Software. This warranty is expressly in lieu of any oral or implied warranties, including without limitation any implied warranty of merchantability or fitness for a particular purpose. Masimo shall not be liable for any incidental, special or consequential loss, damage or expense directly or indirectly arising from the use or loss of use of any Products or Software. In no event shall Masimo's liability arising from any Product and Software (under contract, warranty, tort, strict liability or other claim) exceed the amount paid by purchaser for the Products giving rise to such claim. The limitations in this section shall not be deemed to preclude any liability that cannot legally be disclaimed by contract.

SALES & END-USER LICENSE AGREEMENT

This document is a legal agreement between you ("purchaser") and Masimo Corporation ("Masimo") for the purchase of this Product ("Product") and a license in the included or embedded Software ("Software") except as otherwise expressly agreed in a separate contract for the acquisition of this Product, the following terms are the entire agreement between the parties regarding your purchase of this Product. If you do not agree to the terms of this agreement, promptly return the entire Product, including all accessories, in their original packages, with your sales receipt to Masimo for a full refund.

RESTRICTIONS

1. Copyright Restrictions: The Software and the accompanying written materials are copyrighted. Unauthorized copying of the Software, including Software that has been modified, merged, or included with other software, or the written materials is expressly forbidden. Purchaser may be held legally responsible for any copyright infringement that is caused or incurred by Purchaser's failure to abide by the terms of this Agreement. Nothing in this License provides any rights beyond those provided by 17 U.S.C. §117.

2. Use Restrictions: Purchaser may physically transfer the Product from one location to another provided that the Software is not copied. Purchaser may not electronically transfer the Software from the Product to any other device. Purchaser may not disclose, publish, translate, release, distribute copies of, modify, adapt, translate, reverse engineer, decompile, disassemble, or create derivative works based on the Software or the written materials.

3. Transfer Restrictions: In no event may Purchaser transfer, assign, rent, lease, sell, or otherwise dispose of the Product or the Software on a temporary basis. Purchaser shall not assign or transfer this License, in whole or in part, by operation of law or otherwise without Masimo's prior written consent; except that the Software and all of Purchaser's rights hereunder shall transfer automatically to any party that legally acquires title to the Product with which this Software is included. Any attempt to assign any rights, duties or obligations arising hereunder other than as set forth in this paragraph shall be void.

4. U.S. Government Rights: If Purchaser is acquiring Software (including the related documentation) on behalf of any part of the United State Government, the following provisions apply: the Software and documentation are deemed to be "commercial software" and "commercial computer software documentation," respectively pursuant to DFAR Section 227.7202 FAR 12.212, as applicable. Any use, modification, reproduction, release, performance, display or disclosure of the Software (including the related documentation) by the U.S. Government or any of its agencies shall be governed solely by the terms of this Agreement and shall be prohibited except to the extent expressly permitted by the terms of this Agreement.

NO IMPLIED LICENSE

Purchase or possession of this product does not carry any express or implied license to use this product with any device that is not an authorized device or separately authorized to use product.

CAUTION: FEDERAL LAW (U.S.A.) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN. For professional use. See instructions for use for full prescribing information, including indications, contraindications, warnings, precautions and adverse events.

The following symbols may appear on the product or product labeling:

SYMBOL	DEFINITION	SYMBOL	DEFINITION	SYMBOL	DEFINITION
 (blue background)	Follow instructions for use		Separate collection for electrical and electronic equipment (WEEE).	Rx ONLY	Caution: Federal law (USA) restricts this device to sale by or on the order of a physician
	Consult instructions for use	LOT	Lot code	CE	Mark of conformity to European Medical Device Directive 93/42/EEC
	Manufacturer	REF	Catalogue number (model number)	EC REP	Authorized representative in the European community
	Date of manufacture YYYY-MM-DD	####	Masimo reference number		Body weight
IPX1	Protection against vertically falling water drops		Storage humidity limitation		Storage temperature range
	Do not discard		Do not use if package is damaged		Keep dry
	Fragile, handle with care	>	Greater than	<	Less than
	Non-sterile		Not made with natural rubber latex		Atmospheric pressure limitation
	Instructions/Directions for Use/Manuals are available in electronic format @ http://www.Masimo.com/TechDocs Note: eIFU is not available for CE mark countries.				

Patents: <http://www.masimo.com/patents.htm>

Masimo, SET, Root and  are federally registered trademarks of Masimo Corporation.

Iris, Iris Gateway and Patient SafetyNet are trademarks of Masimo Corporation.

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© 2017 Masimo Corporation
EU Authorized Representative for Masimo Corporation:
MDSS GmbH