

Addendum, Messages

This Addendum applies to the following:

- **Operator's Manual, Rad-57™ and Pronto®**
32900/5052H-1211 and 33664/5813D-0111
- **Software Version V2003 and newer on the Rad-57™ and Pronto®**

Alert Messages

The following additional alert messages applicable to sensors and patient cables will be displayed on Rad-57 and Pronto devices:

Display	Reason	Solution
<i>CbL</i> <i>SEn</i>	Check Sensor Connection.	Unplug and plug in sensor to the sensor port. Ensure sensor connector to the instrument is secure.
<i>rPL</i> <i>CbL</i> (Blinks three times and returns to monitoring screen)	*Patient Cable Life Near Expiration.	Replace the patient cable.
<i>rPL</i> <i>SEn</i> (Blinks three times and returns to monitoring screen)	*Sensor Life Near Expiration.	Replace the sensor.
<i>rPL</i> <i>AdH</i> (Blinks three times and returns to monitoring screen)	*Adhesive Sensor Life Near Expiration.	Replace the adhesive sensor.

*Upon connecting a cable or sensor that is near the end of its active patient monitoring life, the device will display the Life Near Expiration message by blinking rPL CbL, rPL SEn and rPL AdH three times. No additional Life Near Expiration message will appear unless a new connection is established. It is recommended to switch to a new sensor or cable whenever this alert message is displayed.



37206/LAB-8298A 0614