
# Masimo® Forehead Sensor Headband

Images	2
<b>en</b> English	3
<b>fr</b> French	4
<b>de</b> German	5
<b>it</b> Italian	6
<b>es</b> Spanish	7
<b>sv</b> Swedish	8
<b>nl</b> Dutch	9
<b>da</b> Danish	10
<b>pt</b> Portuguese	11
<b>zh</b> Chinese	12
<b>ja</b> Japanese	13
<b>fi</b> Finnish	14
<b>no</b> Norwegian	15
<b>cs</b> Czech	16
<b>hu</b> Hungarian	17
<b>pl</b> Polish	18
<b>ro</b> Romanian	19
<b>sk</b> Slovak	20
<b>tr</b> Turkish	21
<b>el</b> Greek	22
<b>ru</b> Russian	23
<b>ar</b> Arabic	24
<b>fa</b> Farsi	25

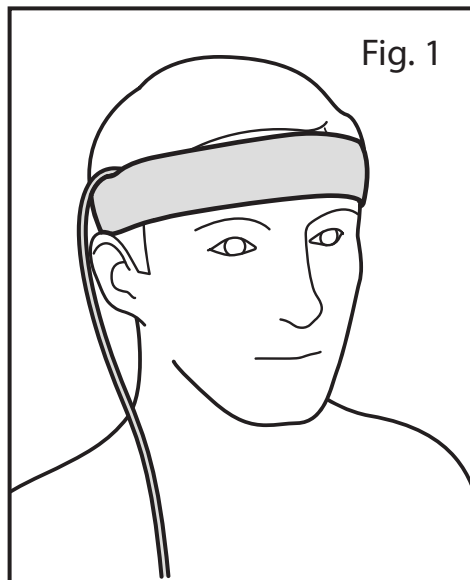# Masimo® Forehead Sensor Headband

	-20°C - +60°C
	10% - 95%

## DIRECTIONS FOR USE

### DESCRIPTION

The Masimo Forehead Sensor Headband is indicated for use with the TFA-I disposable transreflectance sensors and the TF-I reusable transreflectance sensors.

### WARNINGS

- The site must be checked frequently or per clinical protocol to ensure adequate adhesion, circulation, skin integrity and correct optical alignment.
- Exercise extreme caution with poorly perfused patients; skin erosion and pressure necrosis can be caused when the sensor is not frequently moved. Assess site as frequently as every (1) hour with poorly perfused patients and move the sensor if there are signs of tissue ischemia.
- During low perfusion, the sensor site needs to be assessed frequently for signs of tissue ischemia, which can lead to pressure necrosis.
- Do not use tape to secure the sensor to the site; this can restrict blood flow and cause inaccurate readings. Use of additional tape can cause skin damage, and/or pressure necrosis or damage the sensor.
- Sensors applied too tightly or that become tight due to edema will cause inaccurate readings and can cause pressure necrosis
- Misapplied sensors or sensors that become partially dislodged may cause incorrect measurements.

### INSTRUCTIONS

1. Following the instructions that are included with the sensor, apply the sensor to the forehead. Sensor should be just above the eyebrow in line with the center of the eye (pupil).
2. Apply the headband. Headband should be secure enough to apply slight pressure to the sensor and should completely cover the sensor (Fig. 1).
3. The cable should be routed up and over the headband so the cable does not apply pressure to the skin and is not pulling on the sensor.

The following symbols may appear on the product or product labeling:

SYMBOL	DEFINITION	SYMBOL	DEFINITION	SYMBOL	DEFINITION
	Mark of conformity to European medical device directive 93/42/EEC		Separate collection for electrical and electronic equipment (WEEE).	<b>Rx ONLY</b>	<b>Caution:</b> Federal (USA) law restricts this device to sale by or on the order of a physician.
	Consult instructions for use		Atmospheric pressure limitation		Follow instructions for use
	Manufacturer	<b>REF</b>	Catalogue number (model number)	<b>EC REP</b>	Authorized representative in the European community
	Date of manufacture YYYY-MM-DD	<b>####</b>	Masimo reference number		Fragile, handle with care
	Single patient use only		Storage humidity limitation		Storage temperature range
	Non-Sterile		Do not use if package is damaged		Keep dry
	Not made with natural rubber latex		Instructions/Directions for Use/Manuals are available in electronic format @ <a href="http://www.Masimo.com/TechDocs">http://www.Masimo.com/TechDocs</a> <b>Note: eFU is not available in all countries</b>		

Patents: <http://www.masimo.com/patents.htm>

Masimo, TF-I and  are federally registered trademarks of Masimo Corporation.

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© 2019 Masimo Corporation
EU Authorized Representative for Masimo Corporation:
MDSS GmbH