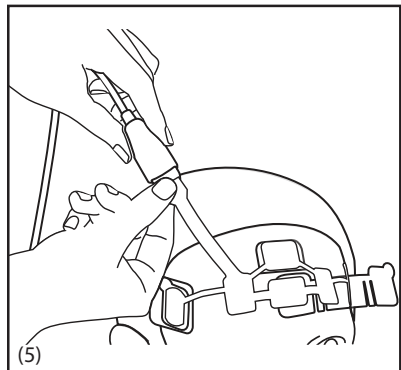
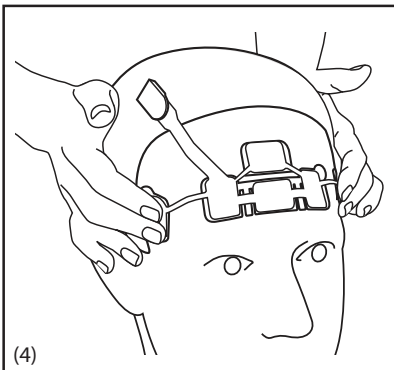
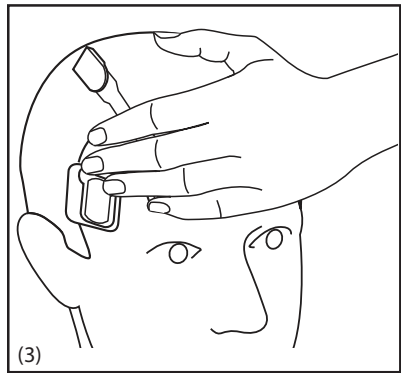
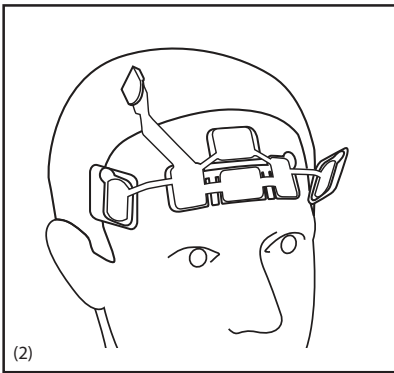
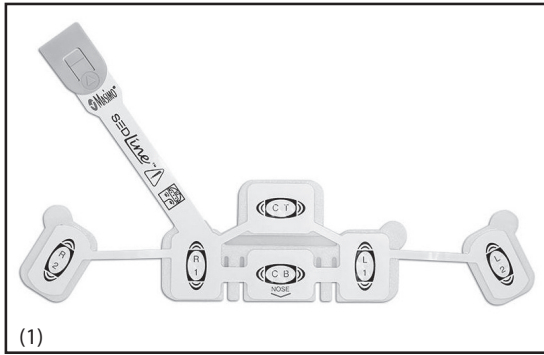
SedLine® Sensor

| | |
|----------------------|-------|
| Images | 2 |
| en English | 3-4 |
| fr French | 5-6 |
| de German | 7-8 |
| it Italian | 9-10 |
| es Spanish | 11-12 |
| sv Swedish | 13-14 |
| nl Dutch | 15-16 |
| da Danish | 17-18 |
| pt Portuguese | 19-20 |
| fi Finnish | 21-22 |
| zh Chinese | 23-24 |
| ja Japanese | 25-26 |
| ko Korean | 27-28 |
| no Norwegian | 29-30 |
| cs Czech | 31-32 |
| hu Hungarian | 33-34 |
| pl Polish | 35-36 |
| ro Romanian | 37-38 |
| sk Slovak | 39-40 |
| tr Turkish | 41-42 |
| el Greek | 43-44 |
| ru Russian | 45-47 |
| ar Arabic | 49-48 |
| fa Farsi | 51-50 |

DIRECTIONS FOR USE

Single Patient Use Only

Not made with natural rubber latex

Non-sterile

Leave on for no more than 24 hours

DESCRIPTION

The SedLine Sensor (1) is comprised of 6 gelled electrodes, 4 active channels (R1, R2, L1, L2), one reference channel (CT), and one ground channel (CB). The Sensor is a single-use, non-sterile product that does not contain natural rubber latex.

Refer to the SedLine Operator's Manual for additional information including Warnings and Cautions.

INDICATIONS

The Sensor electrodes are applied directly to the patient's skin to enable recording of electrophysiologic signals (such as EEG).

CONTRAINDICATIONS

This device is not intended for use in children less than 18 years of age.

WARNINGS, CAUTIONS AND NOTES

- **Warning:** If skin irritation is noticed, discontinue use of the Sensor.
- **Warning:** To avoid cross contamination, only use Masimo single-use sensor on the same patient.
- **Warning:** Do not place electrodes between surgical site and the electro-surgical return electrode. Doing so increases the risk of burns in case of a defect in the electro-surgical return electrode.
- **Caution:** Federal law restricts this device to sale by or on the order of a physician.
- **Caution:** For professional use. See instructions for use for full prescribing information, including indications, contraindications, warnings, precautions and adverse events.
- **Caution:** Use the Sensor immediately after opening to prevent gel from drying out.
- **Caution:** Do not use the Sensor if pouch or product is defective.
- **Caution:** Do not allow electrode gel to come into contact with the SEDLine Sensor connector as this may cause impedance problems and inaccurate PSI readings.
- **Caution:** Each Sensor is approved for use up to 24 hours. If monitoring in excess of this period is desired, replace the Sensor every 24 hours.
- **Caution:** SedLine is designed to operate with the sensor. Simultaneous use with other electrodes on the forehead is not recommended.
- **Caution:** Do not use scissors to open pouch as this may cause damage to the Sensor.
- **Caution:** Do not cut the Sensor as this may result in improper operation.
- **Caution:** Applying excessive pressure to electrodes may lead to decreased signal quality, decreased PSI reliability, and poor adhesion.
- **Caution:** The conductive parts of electrodes, sensors, or connectors should not contact other conductive parts including earth ground.
- Do not attempt to reprocess, recondition or recycle any Masimo sensors or patient cables as these processes may damage the electrical components, potentially leading to patient harm.

INSTRUCTIONS

Connecting the Sensor

To connect the Sensor

1. Prep skin with alcohol and dry before applying Sensor.
2. Peel the release liner from the center foam and discard.
3. Center CB/CT electrodes on forehead above nose (2). Press the white lining around the electrodes firmly onto the forehead with palm of hand to secure the Sensor (3). Do not press directly on the electrode, otherwise the gel may leak out.
4. Apply R2/L2 in hairless area just above temple (4).
5. Apply R1/L1 to either side of CB.
6. Align the white end of the Patient Cable with the arrow on the Sensor connector (5).
7. Push together until the Sensor stiffener latch clicks into place.
8. Ensure the monitor is connected and powered up.
9. If any of the electrodes are displayed as red, yellow, or blue, push and wiggle the white lining around the electrodes. Do not press directly on the electrode, otherwise the gel may leak out.
10. For additional details, see the SedLine Operator's Manual.

Disconnecting the Sensor

To disconnect the Sensor, gently pull the Sensor connector out of the Patient Cable connector.

Sensor Removal

To remove the sensor from the patient's forehead, pull each electrode tab gently away from the skin.

If the sensor is difficult to remove, use alcohol to assist in removal.

Note: The Sensor is not reusable. Dispose of the sensor according to local laws.

WARRANTY

Masimo warrants to the initial buyer only that this product, when used in accordance with the directions provided with the Products by Masimo, will be free of defects in materials and workmanship for a period of six (6) months. Single use products are warranted for single patient use only.

The foregoing is the sole and exclusive warranty applicable to the products sold by Masimo to buyer. Masimo expressly disclaims all other oral, express or implied warranties, including without limitation any warranties of merchantability or fitness for particular purpose. Masimo's sole obligation and buyer's exclusive remedy for breach of any warranty shall be, at Masimo's option, to repair or replace the product.

WARRANTY EXCLUSIONS

This warranty does not extend to any product that has been used in violation of the operating instructions supplied with the product, or has been subject to misuse, neglect, accident or externally created damage. This warranty does not extend to any product that has been connected to any unintended instrument or system, has been modified, or has been disassembled or reassembled. This warranty does not extend to sensors or patient cables that have been reprocessed, reconditioned or recycled.

In no event shall Masimo be liable to buyer or any other person for any incidental, indirect, special or consequential damages (including without limitation lost profits), even if advised of the possibility thereof. In no event shall Masimo's liability arising from any products sold to buyer (under a contract, warranty, tort or other claim) exceed the amount paid by buyer for the lot of product(s) involved in such claim. In no event shall Masimo be liable for any damages associated a product that has been reprocessed, reconditioned or recycled. The limitations in this section shall not be deemed to preclude any liability that, under applicable products liability law, cannot legally be precluded by contract.

NO IMPLIED LICENSE

This single-patient use sensor is licensed to you under the patents owned by Masimo for single-patient use only. By acceptance or use of this product, you acknowledge and agree that no license is granted for use of this product with more than a single patient.

SYMBOLS

If you encounter any serious incident with product, please notify the competent authority in your country and the manufacturer. The following symbols may appear on the product or product labeling:

| SYMBOL | DEFINITION | SYMBOL | DEFINITION | SYMBOL | DEFINITION |
|--|---|--|--|---|---|
|  | Follow instructions for use |  | Separate collection for electrical and electronic equipment (WEEE) | Rx ONLY | Caution: Federal law (USA) restricts this device to sale by or on the order of a physician |
|  | Consult instructions for use | LOT | Lot code | CE | Mark of conformity to European Medical Device Directive 93/42/EEC |
|  | Manufacturer | REF | Catalogue number (model number) | EC REP | Authorized representative in the European community |
|  | Date of manufacture YYYY-MM-DD | ### | Masimo reference number |  | Body weight |
|  | Use -by YYYY-MM-DD |  | Leave on for no more than 24 hours |  | Storage temperature range |
|  | Do not re-use/ Single patient use only | > | Greater than |  | Keep dry |
|  | Non-Sterile | < | Less than |  | Do not use if package is damaged and consult instructions for use |
|  | Not made with natural rubber latex |  | Caution |  | Atmospheric pressure limitation |
|  | Fragile, handle with care | UDI | Unique device identifier |  | Storage humidity limitation |
|  | Single patient - multiple use | MD | Medical device |  | Instructions/Directions for Use/Manuals are available in electronic format @ http://www.Masimo.com/TechDocs Note: eFU is not available in all countries. |

Adhesive Sensors are licensed to purchasers under the foregoing patents for Single-Patient Use Only.

Patents: <http://www.masimo.com/patents.htm>

Masimo and SedLine are federally registered trademarks of the Masimo Corporation.

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© 2020 Masimo Corporation
EU Authorized Representative for Masimo Corporation:
MDSS GmbH