

# Addendum Rad-5<sup>®</sup>/5v<sup>®</sup>: Alarms and Messages

---

This addendum provides updates to the following:

- Operator's Manual, Rad-5/5v Signal Extraction Pulse Oximeter<sup>®</sup> 33606/LAB-5724D 1209 and equivalent translations

## Messages

The device will indicate other data or system errors. Message conditions for the device follow:

Display Message	Type	Solution
nO CbL	No cable connected	Connect cable
Inc CbL	Incompatible cable	Replace cable
rPL CbL	Defective or unrecognized cable	Replace cable
Temporarily blinking message: rPL CbL	Cable life expired	Replace cable as soon as possible
nO SEn	No sensor connected	Connect sensor
Inc SEn	Incompatible sensor	Replace sensor
rPL SEn	Unrecognized or defective sensor	Replace sensor
Temporarily blinking message: rPL SEn	Sensor life expired	Replace sensor as soon as possible
nO AdH	No adhesive sensor connected	Connect adhesive sensor

<b>Display Message</b>	<b>Type</b>	<b>Solution</b>
Inc AdH	Incompatible adhesive sensor	Replace adhesive sensor
rPL AdH	Defective or unrecognized adhesive sensor	Replace adhesive sensor
Temporarily blinking message: rPL AdH	Adhesive sensor life expired	Replace adhesive sensor as soon as possible
rPL AdH	Defective or unrecognized adhesive sensor	Replace adhesive sensor

38043/LAB-9026A 1015