
# RAM® Dual Cable Series







## RD RAM rainbow SET®

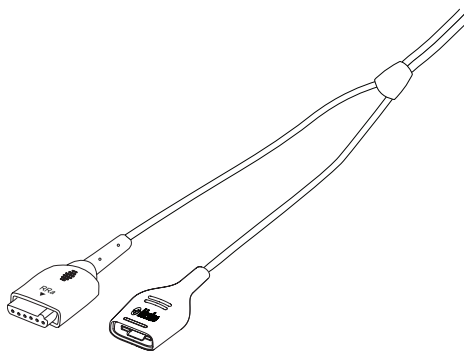Images	2
<b>en</b> English	3-5
<b>fr</b> French	6-8
<b>de</b> German	9-11
<b>it</b> Italian	12-14
<b>es</b> Spanish	15-17
<b>sv</b> Swedish	18-20
<b>nl</b> Dutch	21-23
<b>da</b> Danish	24-26
<b>pt</b> Portuguese	27-29
<b>zh</b> Chinese	30-32
<b>ja</b> Japanese	33-35
<b>fi</b> Finnish	36-38
<b>no</b> Norwegian	39-41
<b>cs</b> Czech	42-44
<b>hu</b> Hungarian	45-47
<b>pl</b> Polish	48-50
<b>ro</b> Romanian	51-53
<b>sk</b> Slovak	54-56
<b>tr</b> Turkish	57-59
<b>el</b> Greek	60-62
<b>ru</b> Russian	63-65
<b>ar</b> Arabic	68-66
<b>fa</b> Farsi	71-69

# RAM® Dual Cable Series

## RD RAM rainbow SET®

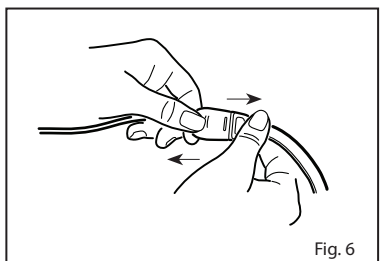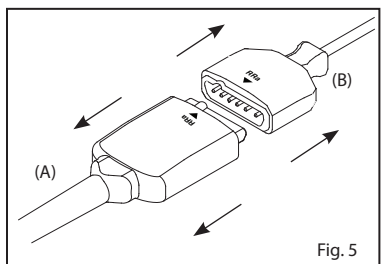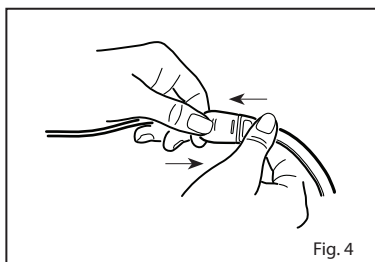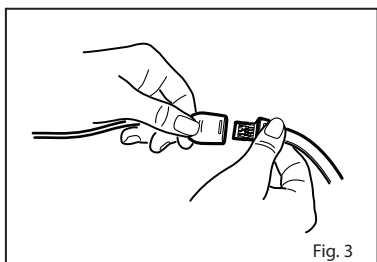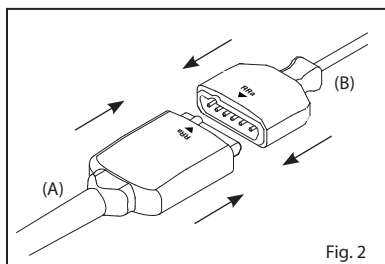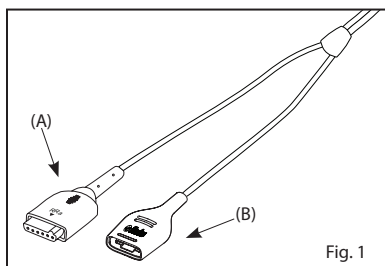# RAM<sup>®</sup> Dual Cable Series

## RD RAM rainbow SET<sup>®</sup>

en

### DIRECTIONS FOR USE

Reusable

 Not made with natural rubber latex

 Non-Sterile

#### INDICATIONS

The RAM<sup>®</sup> Dual Cable Series have the same indications for use as the applicable sensors. Please see the compatible sensor's directions for use for indications for use and prescribed information.

#### DESCRIPTION

The RAM Dual Cable RD rainbow SET<sup>®</sup> cables are for use with the rainbow Acoustic<sup>®</sup> Respiration Sensors (RAS) and RD SET<sup>®</sup> sensors.

Refer to Fig. 1. The RAM Dual Cable RC provides two (2) types of sensor connections:

1. (A) A connection to the Acoustic Respiration Sensor (RAS) and
2. (B) A connection to the RD SET sensor.

The RAM Dual Cable series have been verified using Masimo<sup>®</sup> SET<sup>®</sup> Technology.

#### WARNINGS, CAUTIONS AND NOTES

- Always refer to the oximeter module Operator's Manual for complete instructions or additional instructions.
- Ensure the cable is physically intact, with no broken or frayed wires or damaged parts. Visually inspect the cable and discard if cracks or signs of deterioration are found.
- All sensors and cables are designed for use with specific monitors. Verify the compatibility of the monitor, cable and sensor before use, otherwise degraded performance and/or patient injury can result.
- Carefully route patient cables to reduce the possibility of patient entanglement or strangulation.
- Failure to properly connect the sensor or the oximeter module to the cable will result in intermittent readings, inaccurate results, or no reading.
- To avoid damage to the cable, always hold it by the connector rather than the cable when connecting or disconnecting either end.
- To prevent damage, do not soak or immerse the sensor or cable in any liquid solution. Do not attempt to sterilize the cable or sensor.
- Do not attempt to reprocess, recondition or recycle Masimo sensors or patient cables as these processes may damage the electrical components, potentially leading to patient harm.
- **Caution:** Replace the cable when a replace cable message is displayed, or when a low SIQ message is consistently displayed while monitoring consecutive patients after completing the low SIQ troubleshooting steps identified in the monitoring device operator's manual.
- **Note:** The cable is provided with X-Cal<sup>®</sup> technology to minimize the risk of inaccurate readings and unanticipated loss of patient monitoring. The cable will provide up to 17,520 hours of patient monitoring time. Replace the cable when the patient monitoring time is exhausted. Some legacy monitoring devices do not distinguish between the cable and the sensor. In these devices, a replace sensor or similar message could pertain either to the sensor or the cable.

#### INSTRUCTIONS

**NOTE:** Connect the RD rainbow SET RAM Dual Cable to the rainbow Acoustic Monitoring<sup>®</sup> enabled device before connecting the sensors.

##### A. Connecting to the rainbow Acoustic Monitoring Enabled Device

1. Connect the device connector of the patient cable to the mating connector of the device.
2. Ensure the patient cable connector is fully engaged with the device. A tactile or audible click of connection should be heard.

##### B. Connecting the RD rainbow SET RAM Dual Cable to the Sensors

###### Connect to a RAS sensor:

1. Refer to Fig. 2. Orient the arrow on the RAM Dual Cable acoustic connector (A) with the arrow on the RAS sensor (B).
2. Insert the sensor connector completely into the acoustic connector until it locks in place.

###### Connect to a RD SET disposable sensor:

1. Refer to Fig. 3. Orient the sensor's connector tab with the "shiny" contacts facing up. Orient the cable with the color bar and finger grips facing up.
2. Refer to Fig. 4. Insert the sensor tab into the cable until there is a tactile or audible click of connection. Gently tug on the connectors to ensure a positive contact.

##### C. Disconnecting from the Sensors

**NOTE:** Pull on the connectors only, do not pull the cables.

###### Disconnect from the RAS sensor:

1. Refer to Fig. 5. Hold the center of the RAM Dual Cable RC connector (A) and the acoustic respiration sensor connector (B).
2. Pull the connectors apart to disconnect.

###### Disconnect from the RD SET disposable sensor:

1. Refer to Fig. 6. Pull firmly on the sensor connector to remove it from the patient cable.

**NOTE:** The RAM Dual Cable RD RAM rainbow SET cable can be connected/disconnected while the device is operational.

---

## CLEANING

1. Remove the sensors from the patient and disconnect them from the patient cable.
2. Clean the surface of the cable by wiping it with a 70% isopropyl alcohol pad.
3. Wipe all surfaces of the cable.
4. Saturate another cloth or gauze pad with sterile or distilled water and wipe all surfaces of the cable.
5. Dry the cable by wiping all surfaces with a clean cloth or dry gauze pad.

### CAUTION:

- Do not immerse the connectors on the cable in any liquid solution.
- Do not sterilize by irradiation, steam, autoclave, or ethylene oxide.
- Do not clean with chemicals not approved above.

---

## COMPATIBILITY

This cable is intended for use only with rainbow Acoustic Monitoring enabled devices containing Masimo rainbow SET technology or pulse oximetry monitors licensed to use Acoustic Respiration compatible cables and sensors. Each cable is designed to operate correctly only on the pulse oximetry systems from the original device manufacturer. Use of this cable with other devices may result in no or improper performance.

For Compatibility Information Reference: [www.Masimo.com](http://www.Masimo.com)

---

## ENVIRONMENTALS

Operating Temperature	41°F to 104°F (5°C to 40°C)
Storage Temperature	-40°F to 158°F (-40°C to 70°C)
Relative Humidity	10% to 95% non-condensing

---

## WARRANTY

Masimo warrants to the initial buyer only that this product, when used in accordance with the directions provided with the Products by Masimo, will be free of defects in materials and workmanship for a period of six (6) months. Single use products are warranted for single patient use only.

THE FOREGOING IS THE SOLE AND EXCLUSIVE WARRANTY APPLICABLE TO THE PRODUCTS SOLD BY MASIMO TO BUYER. MASIMO EXPRESSLY DISCLAIMS ALL OTHER ORAL, EXPRESS OR IMPLIED WARRANTIES, INCLUDING WITHOUT LIMITATION ANY WARRANTIES OF MERCHANTABILITY OR FITNESS FOR PARTICULAR PURPOSE. MASIMO'S SOLE OBLIGATION AND BUYER'S EXCLUSIVE REMEDY FOR BREACH OF ANY WARRANTY SHALL BE, AT MASIMO'S OPTION, TO REPAIR OR REPLACE THE PRODUCT.

---

## WARRANTY EXCLUSIONS

This warranty does not extend to any product that has been used in violation of the operating instructions supplied with the product, or has been subject to misuse, neglect, accident or externally created damage. This warranty does not extend to any product that has been connected to any unintended device or system, has been modified, or has been disassembled or reassembled. This warranty does not extend to sensors or patient cables that have been reprocessed, reconditioned or recycled.

IN NO EVENT SHALL MASIMO BE LIABLE TO BUYER OR ANY OTHER PERSON FOR ANY INCIDENTAL, INDIRECT, SPECIAL OR CONSEQUENTIAL DAMAGES (INCLUDING WITHOUT LIMITATION LOST PROFITS), EVEN IF ADVISED OF THE POSSIBILITY THEREOF. IN NO EVENT SHALL MASIMO'S LIABILITY ARISING FROM ANY PRODUCTS SOLD TO BUYER (UNDER A CONTRACT, WARRANTY, TORT OR OTHER CLAIM) EXCEED THE AMOUNT PAID BY BUYER FOR THE LOT OF PRODUCT(S) INVOLVED IN SUCH CLAIM. IN NO EVENT SHALL MASIMO BE LIABLE FOR ANY DAMAGES ASSOCIATED A PRODUCT THAT HAS BEEN REPROCESSED, RECONDITIONED OR RECYCLED. THE LIMITATIONS IN THIS SECTION SHALL NOT BE DEEMED TO PRECLUDE ANY LIABILITY THAT, UNDER APPLICABLE PRODUCTS LIABILITY LAW, CANNOT LEGALLY BE PRECLUDED BY CONTRACT.

---

## NO IMPLIED LICENSE

Purchase or possession of this RAM Dual Cable RD RAM rainbow SET does not carry any express or implied license to use this cable with any device that is not an authorized device or separately authorized to use RAM Dual Cable Series cables.

**CAUTION: FEDERAL LAW (U.S.A.) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN.**

For professional use. See instructions for use for full prescribing information, including indications, contraindications, warnings, precautions and adverse events.

If you encounter any serious incident with product, please notify the competent authority in your country and the manufacturer.

**The following symbols may appear on the product or product labeling:**

SYMBOL	DEFINITION	SYMBOL	DEFINITION	SYMBOL	DEFINITION
	Follow instructions for use		Separate collection for electrical and electronic equipment (WEEE)	<b>Rx ONLY</b>	<b>Caution:</b> Federal law (USA) restricts this device to sale by or on the order of a physician
	Consult instructions for use		Lot code		Mark of conformity to European Medical Device Directive 93/42/EEC
	Manufacturer		Catalogue number (model number)		Authorized representative in the European community
	Date of manufacture YYYY-MM-DD		Masimo reference number		Body weight
	Caution		Storage humidity limitation		Storage temperature range
	Do not discard		Do not use if package is damaged and consult instructions for use		Keep dry
	Fragile, handle with care		Greater than		Less than
	Non-sterile		Not made with natural rubber latex		Atmospheric pressure limitation
	Medical device		Unique device identifier		Instructions/Directions for Use/Manuals are available in electronic format @ <a href="http://www.Masimo.com/TechDocs">http://www.Masimo.com/TechDocs</a> <b>Note: eIFU is not available in all countries.</b>
	Importer		Distributor		

Patents: <http://www.masimo.com/patents.htm>

Masimo, SET, rainbow, rainbow Acoustic Monitoring, rainbow Acoustic, rainbow SET, RD SET, RD rainbow SET, X-Cal, RAM, and RRA are federally registered trademarks of Masimo Corporation.

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© 2021 Masimo Corporation
MDSS GmbH