# RD rainbow SET™

## Adhesive Replacement Tapes

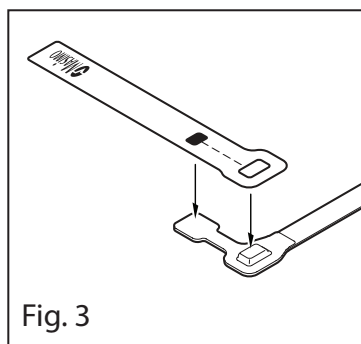
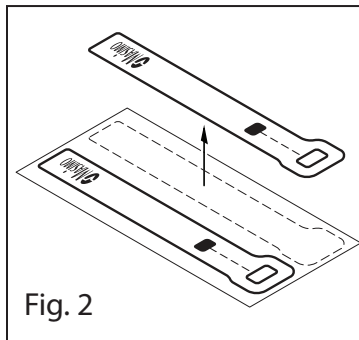
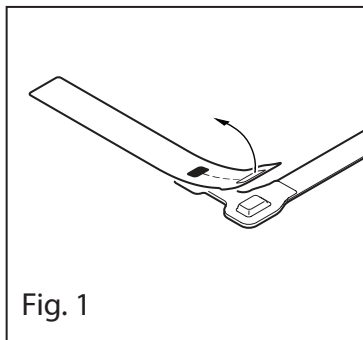
Images	2
<b>en</b> English	3
<b>fr</b> French	4
<b>de</b> German	5
<b>it</b> Italian	6
<b>es</b> Spanish	7
<b>sv</b> Swedish	8
<b>nl</b> Dutch	9
<b>da</b> Danish	10
<b>pt</b> Portuguese	11
<b>zh</b> Chinese	12
<b>ja</b> Japanese	13
<b>fi</b> Finnish	14
<b>no</b> Norwegian	15
<b>cs</b> Czech	16
<b>hu</b> Hungarian	17
<b>pl</b> Polish	18
<b>ro</b> Romanian	19
<b>sk</b> Slovak	20
<b>tr</b> Turkish	21
<b>el</b> Greek	22
<b>ru</b> Russian	23
<b>ko</b> Korean	24
<b>ar</b> Arabic	25

# RD rainbow SET™

## Adhesive Replacement Tapes

# RD rainbow SET™

## Adhesive Replacement Tapes

en

### DIRECTIONS FOR USE

Single patient use only

Not made with natural rubber latex

Non-sterile

#### DESCRIPTION

The RD rainbow SET™ adhesive replacement tapes are used to replace the attachment tape portion of the RD rainbow SET infant (Inf) and neonatal (Neo) adhesive sensors in the event that the tape becomes soiled or torn prior to the end of sensor life.

#### INSTRUCTIONS

1. Remove the existing tape and discard (Fig. 1).
2. Remove the replacement tape from the release liner (Fig. 2).
3. Position the replacement tape over the sensor and press in place (Fig. 3).

The following symbols may appear on the product or product labeling:

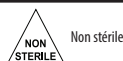
SYMBOL	DEFINITION	SYMBOL	DEFINITION	SYMBOL	DEFINITION
	Consult instructions for use		Do not use if package is damaged		Pediatric patient
	Follow instructions for use		Masimo reference number		Storage humidity limitation
	Manufacturer		Mark of Conformity to European Medical Device Directive 93/42/EEC		Storage temperature range
	Use-by date YYYY-MM-DD		Authorized representative in the European community		Keep dry
	Do not re-use/Single patient use only		Lot code		Atmospheric pressure limitation
	Non-Sterile		Catalogue number (model number)		Caution
	Not made with natural rubber latex	<b>Rx ONLY</b>	<b>Caution:</b> Federal (USA) law restricts this device to sale by or on the order of a physician.		
	Instructions/Directions for Use/Manuals are available in electronic format @ <a href="http://www.Masimo.com/TechDocs">http://www.Masimo.com/TechDocs</a> <b>Note: eIFU is not available for CE mark countries</b>				

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© 2017 Masimo Corporation
Masimo Corporation
EU Authorized Representative for Masimo Corporation:
MDSS GmbH