Adhesive Pads

For use with Masimo® TF-I® Reusable Sensors

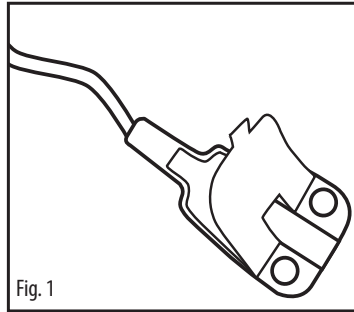
Images	2
en English	3
fr French	4
de German	5
it Italian	6
es Spanish	7
sv Swedish	8
nl Dutch	9
da Danish	10
pt Portuguese	11
zh Chinese	12
ja Japanese	13
fi Finnish	14
no Norwegian	15
cs Czech	16
hu Hungarian	17
pl Polish	18
ro Romanian	19
sk Slovak	20
tr Turkish	21
el Greek	22
ru Russian	23
ko Korean	24
ar Arabic	25

Adhesive Pads

For use with Masimo® TF-I® Reusable Sensors

Adhesive Pads

For use with Masimo® TF-I® Reusable Sensors

en

DIRECTIONS FOR USE

 Single patient use only

 Not made with natural rubber latex

 Non-sterile

Prior to use, the user should read and understand the Operator's Manual for the device and the sensors.

DESCRIPTION

The disposable transreflectance adhesive pads are double sided adhesive pads used to apply adhesive to the reusable transreflectance sensor (TF-I).

INSTRUCTIONS

WARNING: Do not use adhesive pads if the patient exhibits allergic reactions to adhesive tape.

1. Ensure the sensor surface contacting patient skin is dry, clean, and free of debris and oil.
2. Refer to **Fig. 1**. For added Sensor stability, place one adhesive pad on sensor surface to contact patient skin, being careful to align the pad such that it does not cover sensor lenses. Once pad is securely placed, remove top release liner by pulling from release liner tab.

The following symbols may appear on the product or product labeling:

SYMBOL	DEFINITION	SYMBOL	DEFINITION	SYMBOL	DEFINITION
	Follow instructions for use		Not made with natural rubber latex	Rx ONLY	Caution: Federal law (USA) restricts this device to sale by or on the order of a physician
	Consult instructions for use		Lot code		Mark of conformity to European Medical Device Directive 93/4/EEC
	Manufacturer		Catalogue number (model number)		Authorized representative in the European community
	Date of manufacture YYYY-MM-DD		Masimo reference number		Body weight
	Use by YYYY-MM-DD		Storage humidity limitation		Storage temperature range
	Do not re-use/Single patient use only		Do not use if package is damaged		Keep dry
	Non-Sterile		Instructions/Directions for Use/Manuals are available in electronic format @ http://www.Masimo.com/TechDocs Note: eIFU is not available for CE mark countries.		

Patents: <http://www.masimo.com/patents.htm>

Masimo, SET,  and TF-I are federally registered trademarks of Masimo Corporation.

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© 2019 Masimo Corporation