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제품명: UniView Software

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[Iris Gateway]

1. Application Configuration (Device & Location) 메뉴

```

Iris Gateway Appliance Configuration Tool
Please select from one of the following options
(Application Configuration Device & Location Menu)

1. Modify Device Location
2. Modify Location
3. Modify Device Label
4. Add Device
5. Add Location
6. Remove Device
7. Remove Location
8. Display Devices
9. Display Locations

Enter 'exit' to return to the previous menu
Or type 'help #':

```

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```

1. Application Configuration (Device & Location)
2. Application Configuration
3. Server & OS Configuration
4. HL7 Interface Outbound Configuration
5. HL7 Interface Inbound Configuration

```

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```

Enter 'exit' to return to the previous menu

LOC# | Floor | Point of Care | Room | Bed
-----
1      8West  NICU           E789  A

Type the id (1,2) of the location from the above list: 1

1. Floor
2. Point of Care
3. Room
4. Bed

Select the data of the Location to edit: 3

Enter the Room for the Location (Room cannot be empty): E889

Location was successfully modified!
LOC# | Floor | Point of Care | Room | Bed
-----
1      8West  NICU           E889  A

Press enter to return to the main selection menu

```

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```

Enter 'exit' to return to the previous menu
Do you wish to use an existing Floor? (Y/N): y
FLR# | Floor
-----
1      8West
Please choose one of the above Floor id (FLR#): 1
Do you wish to use an existing Point of Care? (Y/N): y
POC# | Point of Care
-----
1      NICU
Please choose one of the above Point of Care areas (POC#): 1
Enter the Room for the Location (Room cannot be empty): E342
Enter the Bed for the Location: A
Location as described by the user:
Floor: 8West
Point of Care: NICU
Room: E342
Bed: A
Is this the desired location to be added? (Y/N): y
Location successfully created!
LOC# | Floor | Point of Care | Room | Bed
-----
1      8West  NICU          E342  A
Press enter to return to the main selection menu

```

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```

Enter 'exit' to return to the previous menu
LOC# | Floor | Point of Care | Room | Bed
-----
1      8West  NICU           E889  A

Type the id (1,2) of the location from the above list: 1

Location to be removed:
Floor: 8West
Point of Care: NICU
Room: E889
Bed: A
Are you sure you want to remove the following location? (Y/N):

```

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

```

Enter 'exit' to return to the previous menu
-----
DEV# | Label          | Serial      | Location {Floor | Point of Care | Room | Bed}| CONNECTED
-----
1    | Instrument1    | 2000008011 | 8West  | NICU    |          | E175  | A    |          | YES
2    | Instrument1234 | 2000001234 | 8West  | NICU    |          | E889  | A    |          | NO
3    | Instrument814  | 200000814  | 8West  | NICU    |          | E889  | A    |          | NO
4    | Instrument816  | 200000816  | 8West  | NICU    |          | E173  | A    |          | NO
-----
Type the id (1,2) to select that device from the above list: 1
-----
Enter the new label for the POC Device.
Spaces and special characters are not allowed: Instrument8011
-----
POC Device was modified successfully!
-----
DEV# | Label          | Serial      | Location {Floor | Point of Care | Room | Bed}| CONNECTED
-----
1    | Instrument8011 | 2000008011 | 8West  | NICU    |          | E175  | A    |          | YES
-----
Press enter to return to the main selection menu

```

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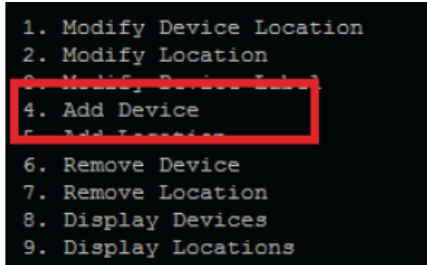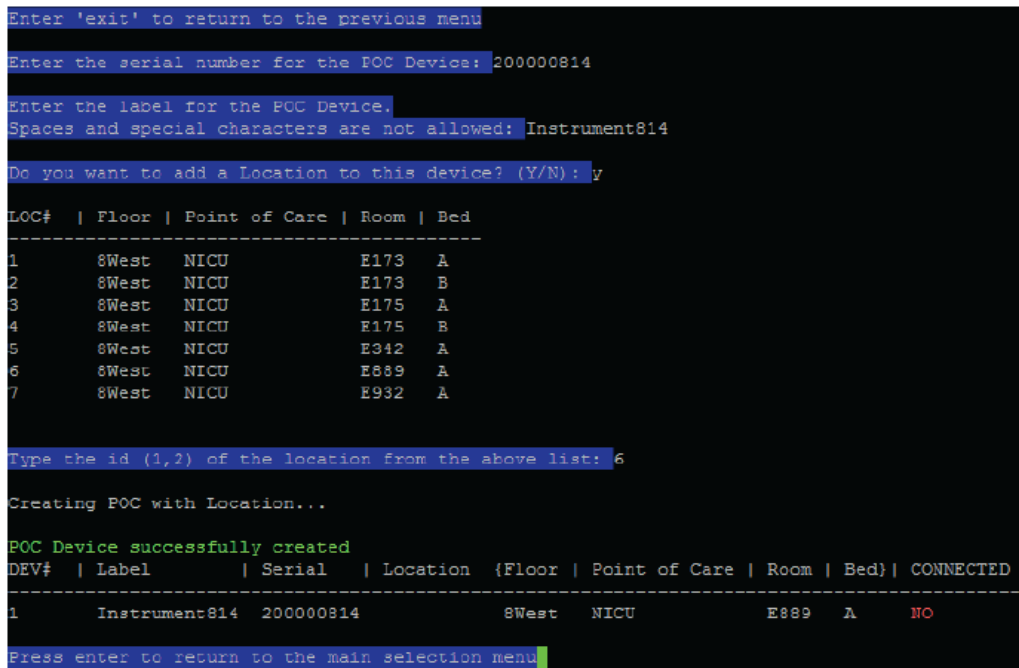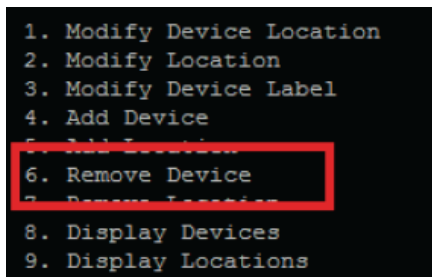
```

Enter 'exit' to return to the previous menu
-----
DEV# | Label          | Serial      | Location {Floor | Point of Care | Room | Bed}| CONNECTED
-----
1    | Instrument1234 | 2000001234 | 8West  NICU    | E889  A    | NO
2    | Instrument8011 | 2000008011 | 8West  NICU    | E175  A    | YES
3    | Instrument814  | 200000814  | 8West  NICU    | E889  A    | NO
4    | Instrument816  | 200000816  | 8West  NICU    | E173  A    | NO
-----
Type the id (1,2) to select that device from the above list: 1
Device to be removed:   POC Device ID: Instrument1234
Are you sure you want to remove the following device? (Y/N): y
POC Device successfully removed!
Press enter to return to the main selection menu
  
```

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2. Application Configuration 메뉴

```

Iris Gateway Appliance Configuration Tool
Please select from one of the following options
(Appliance Configuration, General Menu)

HL7 ADT: Enabled
HL7 Alarm Reporter: Enabled
HL7 Vitals Out: Disabled

1. Check Running Services
2. Restart HL7 Interface ADT & Vitals Out Service
3. Restart HL7 Interface Alarm Reporter Service
4. Restart Iris Gateway Service

Enter 'exit' to return to the previous menu
Select the number of one of the above options

Or type 'help #':
  
```

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```
1. Application Configuration (Device & Location)
2. Application Configuration
3. Server & OS Configuration
4. HL7 Interface Outbound Configuration
5. HL7 Interface Inbound Configuration
```

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```
1. Check Running Services
2. Restart HL7 Interface ADT & Vitals Out Service
3. Restart HL7 Interface Alarm Reporter Service
4. Restart Iris Gateway Service
```

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```
HL7 ADT & Vitals Out service is UP, and was started on 2015-09-25 at 12:00:48
HL7 Alarm Reporter service is UP, and was started on 2015-09-25 at 12:00:48
Press enter to continue to the main menu
```

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```
*****WARNING*****
Restarting the HL7 ADT & Vitals Out service will interrupt the service
Instrument Parameter data will not be sent out while the service is being restarted
Make sure that hospital administrators are informed of any HL7 interruption.

Are you sure you want to restart the HL7 ADT & Vitals Out service? (Y/N): y
Restarting HL7 ADT & Vitals Out ...
HL7 Vitals Out & ADT successfully restarted

Press enter to continue to the main menu
```

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```
*****WARNING*****
Restarting the HL7 Alarm Reporter service will interrupt the service
Patient and Instrument Alarms not be sent out while the service is being restarted
Make sure that hospital administrators are informed of any HL7 Alarm interruption.

Are you sure you want to restart the HL7 Alarm Reporter? (Y/N): y
Restarting HL7 Alarm Reporter ...
HL7 Alarm Reporter successfully restarted

Press enter to continue to the main menu
```

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```
*****WARNING*****
Restarting the Iris Gateway service will interrupt all services
All Instruments will disconnect while the server is not on.
Instrument Parameter data will not be sent out while the service is being restarted
Patient and Instrument Alarms not be sent out while the service is being restarted
Make sure that hospital administrators are informed of any Iris Gateway interruption.

Are you sure you want to restart the Iris Gateway? (Y/N): y
Restarting Iris Gateway ...
```

3. Server & OS Configuration 메뉴

```
Iris Gateway Appliance Configuration Tool
Please select from one of the following options
(Server & OS Configuration Menu)

Current User: root
Current Date & Time: 09 09 2015, 08:32
Current IP Address: 10.1.104.220

1. Shutdown Appliance
2. Restart Appliance
3. Change Network Interface Settings
4. Change Date
5. Change Password

Enter 'exit' to return to the previous menu
Select the number of one of the above options

Or type 'help #':
```

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```
1. Shutdown Appliance
2. Restart Appliance
3. Change Network Interface Settings
4. Change Date
5. Change Password
```

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```

Enter 'exit' at any time to return to the previous menu

*****WARNING*****
Shutting down the Iris Appliance server will stop Iris from running
until the server is turned on manually.
All Instruments will disconnect while the server is not on.
Make sure that hospital administrators are informed of any Iris downtime.

Would you like to shutdown the Iris server now? (Yes/No): yes

Reason for shutting down? (Press enter to leave blank):

```

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```

1. Shutdown Appliance
2. Restart Appliance
3. Change Network Interface Settings
4. Change Date
5. Change Password

```

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```

Enter 'exit' at any time to return to the previous menu

*****WARNING*****
Restarting the Iris Appliance server will stop Iris from running
until the server comes back on.
All Instrument and SafeyNet Views will go down while the server is not on.
Make sure that hospital administrators are informed of any Iris downtime.

Would you like to restart the Iris server now? (Yes/No):

```

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```

Enter 'exit' at any time to return to the previous menu

*****WARNING*****
Changing the Network Interface settings will
require the Network Interface to restart
You will lose connection to your remote session
To reestablish a remote connection, log in to the updated IP Address
All Instruments and SafetyNet Views will
temporarily disconnect during this restart
Enter IP address in dotted-decimal format (ex 192.168.1.120).

Press enter to leave unchanged: 10.1.104.220
Enter subnet mask in dotted decimal format (ex 255.255.255.0).

Press enter to leave unchanged: 255.255.255.0
Enter gateway IP address in dotted decimal format (ex 255.255.255.0).

Press enter to leave unchanged: 10.1.104.254

New IP address: 10.1.104.220
New netmask address: 255.255.255.0
New gateway address: 10.1.104.254

Are you sure you want to use these network settings? (Y/N):

```

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4. HL7 Interface Outbound Configuration 메뉴

```
Iris Gateway Appliance Configuration Tool
Please select from one of the following options
(HL7 Interface Outbound Configuration)

Available HL7 Vitals Out Listeners | HL7 Vitals Out Timers (in secs)
Name          IP address  Port  | Publish Time
HL7MLLPClient 127.0.0.1  : 5555 | 60

Available HL7 Alarm Listeners      | HL7 Alarm Timers (in secs)
Name          IP address  Port  | Keep Alive / Publish Time / Expiration Time
HL7MLLPClient-ACM 127.0.0.1 : 5555 | 60          60          600

1. Add HL7 Vitals Out Listener
2. Add HL7 Alarm Listener
3. Change HL7 Vitals Out Listener IP Address
4. Change HL7 Alarm Listener IP Address
5. Change HL7 Vitals Out port
6. Change HL7 Alarm Listener Port
7. Change Vitals Out Message Frequency
8. Change Alarm Publish Message Frequency
9. Change Unacknowledged Alarm Expiration Time
10. Change HL7 Alarm Reporter Keep Alive Frequency
11. Remove HL7 Vitals Out Listener
12. Remove HL7 Alarm Listener
13. Disable HL7 Vitals Out Service
14. Enable Unassigned HL7 Results Output
15. Disable HL7 Alarm Report Service

Enter 'exit' to return to the previous menu
NOTICE: Pending changes require an HL7 ADT & Vitals Out restart before taking affect
Please restart the appropriate HL7 service(s) before exiting the Iris Gateway Tool

Select the number of one of the above options

Or type 'help #':
```

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```

Iris Gateway Appliance Configuration Tool
Please select from one of the following options
(HL7 Interface Outbound Configuration)

Available HL7 Vitals Out Listeners| HL7 Vitals Out Timers (in secs)
Name IP address Port | Publish Time
EMR1 10.1.104.200 : 4545 | 60

Available HL7 Alarm Listeners | HL7 Alarm Timers (in secs)
Name IP address Port | Keep Alive / Publish Time / Expiration Time
HL7MLLPClient-ACM 127.0.0.1 : 5555 | 60 60 600

1. Add HL7 Vitals Out Listener
2. Add HL7 Alarm Listener
3. Change HL7 Vitals Out Listener IP Address
4. Change HL7 Alarm Listener IP Address
5. Change HL7 Vitals Out port
6. Change HL7 Alarm Listener Port
7. Change Vitals Out Message Frequency
8. Change Alarm Publish Message Frequency
9. Change Unacknowledged Alarm Expiration Time
10. Change HL7 Alarm Reporter Keep Alive Frequency
11. Remove HL7 Vitals Out Listener
12. Remove HL7 Alarm Listener
13. Disable HL7 Vitals Out Service
14. Enable Unassigned HL7 Results Output
15. Disable HL7 Alarm Report Service

Enter 'exit' to return to the previous menu
NOTICE: Pending changes require an HL7 Vitals Out restart before taking affect
Please restart the appropriate HL7 service(s) before exiting the Iris Gateway Tool

Select the number of one of the above options

Or type 'help #':

```

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```

Enter 'exit' at any time to return to the previous menu

HL7 Vitals Out successfully enabled

Press enter to continue to the main menu

```

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```

8. Change Alarm Publish Message Frequency
9. Change Unacknowledged Alarm Expiration Time
10. Change HL7 Alarm Reporter Keep Alive Frequency
11. Remove HL7 Vitals Out Listener
12. Remove HL7 Alarm Listener
13. Disable HL7 Vitals Out Service
14. Enable Unassigned HL7 Results Output
15. Disable HL7 Alarm Report Service

```

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5. HL7 Interface Inbound Configuration 메뉴

```
Iris Gateway Appliance Configuration Tool
Please select from one of the following options
(HL7 Interface Inbound Configuration)

HL7 ADT Port Configuration : 4444

1. Edit the listening port for ADT
2. Disable HL7 ADT service

Enter 'exit' to return to the previous menu
Select the number of one of the above options

Or type 'help #':
```

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```
1. Edit the listening port for ADT
2. Disable HL7 ADT service
```

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```
Enter 'exit' at any time to return to the previous menu

Please enter the new port number (Valid Range 1 - 65535): 3215

New ADT port number of 3215 successfully saved

Press enter to continue to the main menu
```

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